


3M™ Coban™ Self-Adherent Wrap

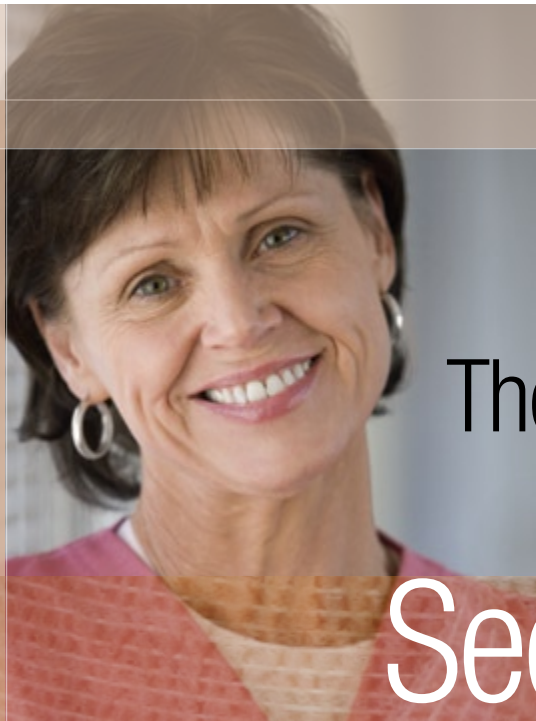

The Brand
You Trust to
Secure, Compress
and Immobilize



3M™ Coban™ Self-Adherent Wrap

For more than 40 years, 3M™ Coban™ Self-Adherent Wrap has been a market leader and a recognized, trusted cohesive wrap for securing dressings and devices, delivering controlled compression, and immobilizing injuries.


Features

- Maintains even compression over time; does not slip
- Provides excellent cohesion – sticks to itself
- Conformable; fits body contours
- Easy to apply and remove; can be re-positioned
- Does not stick to skin, hair, clothing or surgical drapes
- Lightweight, breathable and moisture resistant
- Variety of choices: latex-free, sterile, longer length, color packs

Benefits


- Convenient – no pins, clips or tapes required
- Comfortable for patients
- Leaves no residue on the skin
- Protects primary dressings

Application and Removal

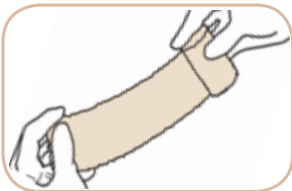

Step 1

Unwind 12 or more inches of Coban wrap from the roll and allow the wrap to relax. *It is important to unwind a sufficient length before beginning to wrap.*

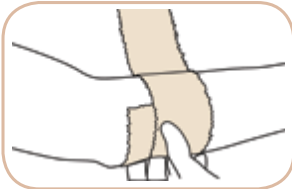

Step 2

Without stretching, apply one full wrap and overlap. Press overlapped area lightly to keep the end in place.



Step 3

Continue wrapping*, applying tension to the wrap only to obtain desired compression.



Step 4

Cut off excess wrap; press end to secure.

* NOTE: *Remember to unwind more Coban wrap from the roll so that the product is not applied with too much tension*

Trusted Performance for a **Variety** of Applications

Suggested Applications



Secural

Secure dressings

- Ideal for patients with fragile skin or where adhesives are not desired
- In difficult-to-dress areas such as the head or fingers and toes
- After blood draws

Secure IVs and other devices

- IV and catheter stabilization
- Sterile drape stabilization in the OR
- Secure tubing, or devices such as 3M™ ColdHot packs



Compression

- Reduce edema as an overwrap for a zinc paste bandage for patients with venous insufficiency
 - *Longer length style for tall and larger patients and patients with lower extremity edema*
- Reduce post-operative swelling, such as
 - *Arthroscopic surgery*
 - *Plastic surgery, liposuction & other cosmetic procedures*



Immobilization

- Overwrap for splints, bivalve and univalve casts
- Support sprains and strains
- Positioning limbs in the OR
- Support post-cast removal

3M™ Coban™ Self-Adherent Wrap

Ordering Information

3M™ Coban™ LF Latex Free Self-Adherent Wrap

| Catalog No. | NDC No. | Size* | Rolls/Case | HCPCS** |
|-----------------------|--------------|------------------------------|------------|---------|
| 2081 | 8333-2081-01 | 1 in. x 5 yd./25 mm x 4,5 m | 30 | A6453 |
| 2082 | 8333-2082-01 | 2 in. x 5 yd./50 mm x 4,5 m | 36 | A6453 |
| 2083 | 8333-2083-01 | 3 in. x 5 yd./75 mm x 4,5 m | 24 | A6454 |
| 2083S (Sterile) | 8333-2083-10 | 3 in. x 5 yd./75 mm x 4,5 m | 24 | A6454 |
| 2084 | 8333-2084-01 | 4 in. x 5 yd./100 mm x 4,5 m | 18 | A6454 |
| 2084S (Sterile) | 8333-2084-10 | 4 in. x 5 yd./100 mm x 4,5 m | 18 | A6454 |
| 2086 | 8333-2086-01 | 6 in. x 5 yd./150 mm x 4,5 m | 12 | A6455 |
| 2086S (Sterile) | 8333-2086-10 | 6 in. x 5 yd./150 mm x 4,5 m | 12 | A6455 |
| 2084L (Longer Length) | 8333-2084-30 | 4 in. x 6.5 yd./100 mm x 6 m | 18 | A6454 |

3M™ Coban™ Self-Adherent Wrap***

| Catalog No. | NDC No. | Size* | Rolls/Case | HCPCS** |
|-----------------------|--------------|------------------------------|------------|---------|
| 1581 | 8333-1581-01 | 1 in. x 5 yd./25 mm x 4,5 m | 30 | A6453 |
| 1582 | 8333-1582-01 | 2 in. x 5 yd./50 mm x 4,5 m | 36 | A6453 |
| 1583 | 8333-1583-01 | 3 in. x 5 yd./75 mm x 4,5 m | 24 | A6454 |
| 1584 | 8333-1584-01 | 4 in. x 5 yd./100 mm x 4,5 m | 18 | A6454 |
| 1586 | 8333-1586-01 | 6 in. x 5 yd./150 mm x 4,5 m | 12 | A6455 |
| 1584L (Longer Length) | 8333-1584-30 | 4 in. x 6.5 yd./100 mm x 6 m | 18 | A6454 |
| 1583S (Sterile) | 8333-1583-10 | 3 in. x 5 yd./75 mm x 4,5 m | 24 | A6454 |
| 1584S (Sterile) | 8333-1584-10 | 4 in. x 5 yd./100 mm x 4,5 m | 18 | A6454 |
| 1586S (Sterile) | 8333-1586-10 | 6 in. x 5 yd./150 mm x 4,5 m | 12 | A6455 |

3M™ Coban™ Self-Adherent Wrap*** (Colors)

| Catalog No. | NDC No. | Size* | Rolls/Case | HCPCS** |
|----------------------|--------------|------------------------------|------------|---------|
| 1581B (Blue) | 8333-1581-02 | 1 in. x 5 yd./25 mm x 4,5 m | 30 | A6453 |
| 1582B (Blue) | 8333-1582-02 | 2 in. x 5 yd./50 mm x 4,5 m | 36 | A6453 |
| 1583B (Blue) | 8333-1583-02 | 3 in. x 5 yd./75 mm x 4,5 m | 24 | A6454 |
| 1583G (Green) | 8333-1583-07 | 3 in. x 5 yd./75 mm x 4,5 m | 24 | A6454 |
| 1583R (Red) | 8333-1583-18 | 3 in. x 5 yd./75 mm x 4,5 m | 24 | A6454 |
| 1583W (White) | 8333-1583-23 | 3 in. x 5 yd./75 mm x 4,5 m | 24 | A6454 |
| 1583Y (Yellow) | 8333-1583-25 | 3 in. x 5 yd./75 mm x 4,5 m | 24 | A6454 |
| 1584B (Blue) | 8333-1584-02 | 4 in. x 5 yd./100 mm x 4,5 m | 18 | A6454 |
| 1583A (Rainbow Pack) | 8333-1583-26 | 3 in. x 5 yd./75 mm x 4,5 m | 12 | A6454 |
| 1583N (Neon Pack) | 8333-1583-14 | 3 in. x 5 yd./75 mm x 4,5 m | 12 | A6454 |



Rainbow Pack



Neon Pack

Note: Coban wrap is available in Custom Procedure Trays. Contact your local tray provider or 3M Health Care Sales Representative.

* Fully-stretched length
 ** Disclaimer: Reimbursement information provided by 3M Health Care and its representatives is intended to provide general information relevant to coding for 3M products. Insurers' reimbursement policies can vary, however, and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level.

*** **CAUTION: These Products Contain Natural Rubber Latex Which May Cause Allergic Reactions.**

To learn more about 3M Skin & Wound Care products, call the 3M Health Care Helpline at 1-800-228-3957, or visit www.3M.com/skinhealth.



Skin & Wound Care Division
3M Health Care
 3M Center, Building 275-4W-02
 St. Paul, MN 55144-1000
 USA
 1-800-228-3957
www.3M.com/healthcare

Medical Division
3M Canada
 Post Office Box 5757
 London, Ontario N6A 4T1
 Canada
 1-800-364-3577

3M and Coban are trademarks of 3M.
 Please recycle.
 Printed in U.S.A.
 © 3M 2009.
 All rights reserved.
 70-2008-5940-6 (59.5) DPI