

# The Many Uses of 3M™ Medipore™ Soft Cloth Products



3M™ Medipore™ H Soft Cloth Surgical Tape  
and 3M™ Tegaderm™ Foam Dressing on Hip

Skin Friendly,  
Soft, Stretchy, Easy-to-Use



3M™ Medipore™ +Pad  
Soft Cloth Adhesive Wound  
Dressing on Knee




3M™ Medipore™ H Soft Cloth Surgical  
Tape on Abdominal Incision



3M™ Medipore™ H Soft Cloth  
Surgical Tape with Catheter Tubing



3M™ Medipore™ +Pad  
Soft Cloth Adhesive Wound  
Dressing on Laminectomy




3M™ Medipore™ Dress-It Pre-Cut  
Dressing Cover with Gauze for  
Compression over Chest Tube Site



**3M**

## Application Tips

### For smooth, easy tearing of 3M™ Medipore™ H Tape

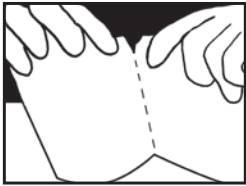


1. Unroll slightly beyond the desired length.



2. Slightly bend the rolled-out segment of tape to identify the perforation lines which are spaced every 2 inches along the entire roll.

3. Grasp the top edge of the tape closest to the perforation line, and **quickly** “tear” the tape apart.


### For easy application of 3M™ Medipore™ Dress-It Pre-Cut Dressing Cover

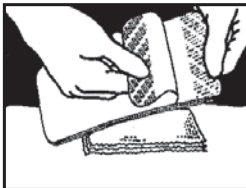


1. Hold the tabbed end as you peel one dressing cover from the pad.



2. Lay the dressing cover over the gauze or area you are covering.

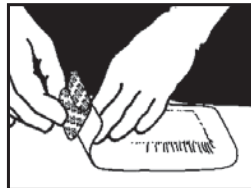

3. Remove the paper tab and secure tape by rubbing or pressing into place.

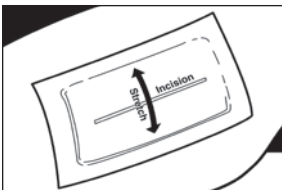

*3M™ Medipore™ H Soft Cloth Surgical Tapes stretch in the cross-direction and on the bias. The direction of the stretch should be considered when securing a dressing or tubing to an area that is at high risk for distention, edema, hematoma formation, or movement.*

## Ordering Information

### 3M™ Medipore™ H Soft Cloth Surgical Tape

Catalog No.	NDC/NHRC No.	Size (in × yd / cm × m)	Rolls/Box	Boxes/Case	HCPCS Code*
<b>2861</b>	8333-2861-01	1 × 10 / 2,5 × 9,1	24	1	A4452
<b>2862</b>	8333-2862-01	2 × 10 / 5 × 9,1	12	1	A4452
<b>2863</b>	8333-2863-01	3 × 10 / 7,6 × 9,1	12	1	A4452
<b>2864</b>	8333-2864-01	4 × 10 / 10,1 × 9,1	12	1	A4452
<b>2866</b>	8333-2866-01	6 × 10 / 15,2 × 9,1	12	1	A4452
<b>2868</b>	8333-2868-01	8 × 10 / 20,3 × 9,1	6	1	A4452
<b>2862S</b>	8333-2862-03	2 × 2 / 5 × 1,8	48	1	A4452
<b>2864S</b>	8333-2864-03	4 × 2 / 10,1 × 1,8	24	1	A4452
<b>2866S</b>	8333-2866-03	6 × 2 / 15,2 × 1,8	16	1	A4452

### 3M™ Medipore™ Dress-It Pre-Cut Dressing Covers

Catalog No.	NDC/NHRC No.	Size (in × in / cm × cm)	Rolls/Box	Boxes/Case	HCPCS Code*
<b>2954</b>	8333-2954-01	3 7/8 × 4 5/8 / 10 × 12	25 (3 sheets/pad)	4	A4452
<b>2955</b>	8333-2955-01	3 7/8 × 7 7/8 / 10 × 20	25 (3 sheets/pad)	4	A4452
<b>2956</b>	8333-2956-01	5 7/8 × 5 7/8 / 15 × 15	25 (3 sheets/pad)	4	A4452
<b>2957</b>	8333-2957-01	5 7/8 × 11 / 15 × 28	25 (3 sheets/pad)	4	A4452
<b>2958</b>	8333-2958-01	7 7/8 × 11 / 20 × 28	25 (3 sheets/pad)	4	A4452

### 3M™ Medipore™ +Pad Soft Cloth Adhesive Wound Dressing

Catalog No.	NDC/NHRC No.	Dressing Size (in × in / cm × cm)	Pad Size (in × in / cm × cm)	Rolls/Box	Boxes/Case	HCPCS Code*
<b>3562</b>	8333-3562-01	2 × 2 3/4 / 5 × 7	1 × 1 1/2 / 2,5 × 4	50	4	A6219
<b>3564</b>	8333-3564-01	2 3/8 × 4 / 6 × 10	1 × 2 3/8 / 2,5 × 6	50	4	A6219
<b>3566</b>	8333-3566-01	3 1/2 × 4 / 9 × 10	1 3/4 × 2 3/8 / 4,5 × 6	25	4	A6219
<b>3568</b>	8333-3568-01	6 × 6 / 15 × 15	4 × 4 / 10 × 10	25	4	A6219
<b>3569</b>	8333-3569-01	3 1/2 × 6 / 9 × 15	1 3/4 × 4 / 4,5 × 10	25	4	A6219
<b>3570</b>	8333-3570-01	3 1/2 × 8 / 9 × 20	1 3/4 × 6 / 4,5 × 15	25	4	A6219
<b>3571</b>	8333-3571-01	3 1/2 × 10 / 9 × 25	1 3/4 × 8 / 4,5 × 20	25	4	A6219
<b>3573</b>	8333-3573-01	3 1/2 × 13 3/4 / 9 × 35	1 3/4 × 11 3/4 / 4,5 × 30	25	4	A6220

To learn more about skin care and the Medipore™ family of products, visit us at [www.3M.com/skinhealth](http://www.3M.com/skinhealth), contact your 3M Skin Health Sales Representative or call the 3M Health Care Customer Helpline at 1-800-228-3957. Outside of the United States, contact the local 3M subsidiary.



**Skin & Wound Care Division**  
**3M Health Care**  
 2510 Conway Ave.  
 St. Paul, MN 55144  
 USA  
 1-800-228-3957  
[www.3M.com/healthcare](http://www.3M.com/healthcare)

**3M Canada**  
 P.O. Box 5757  
 London, Ontario N6A 4T1  
 Canada  
 1-800-364-3577  
[www.3M.com/ca/healthcare](http://www.3M.com/ca/healthcare)

3M, Medipore and Tegaderm are trademarks of 3M.  
 Please recycle.  
 Printed in U.S.A.  
 © 3M 2007, 2012.  
 All rights reserved.  
 70-2009-9556-4

\*Disclaimer: The reimbursement information provided by 3M Health Care and its representatives is intended to provide general information relevant to coding for 3M products. Insurers' reimbursement policies can vary, however, and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level.