
vista 639


LUMBOSACRAL ORTHOSIS

VISTA® 639 LSO 4 PANEL

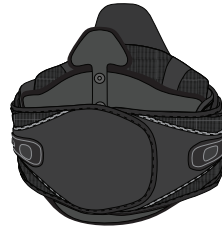ASPEN MEDICAL PRODUCTS
 6481 Oak Canyon, Irvine, CA 92618-5202, USA
 800-295-2776 · 949-681-0200 · aspenmp.com

BELT SIZING FOR VISTA® 627 LUMBAR, 631 LSO LoPro, 637 LSO, 464 TLSO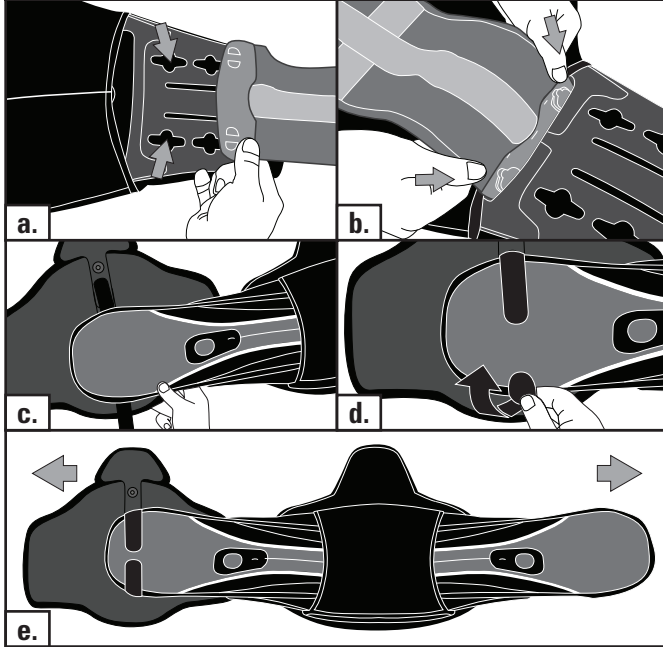

ENGLISH

BELT SIZING

- Fit male rivets of belt through proper sizing adjustment holes in sides of back panel.
- For smaller sizes, slide side panel through sizing adjustment slot and then fit rivets through proper adjustment holes from the under side.
- Align and attach left side panel to rigid anterior panel.
- Apply hook straps to grey loop of side panel.
- Once proper size is achieved, pull taught to lock rivets in place.

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

* The Vista® 627 sizing follows closely to standard pants sizing. Size ranges are only guidelines. Your experiences and personal assessment could make the difference in choosing the correct size. For the Vista® 639 add 2 inches for a better fit.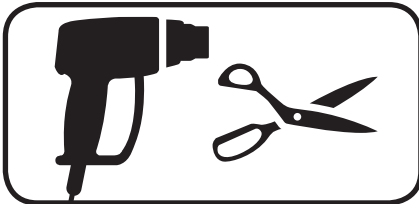

This product may be substantially modified, by an individual with expertise.

For more information, please visit
www.aspenmp.com/index.php/about-us/product-downloads/.

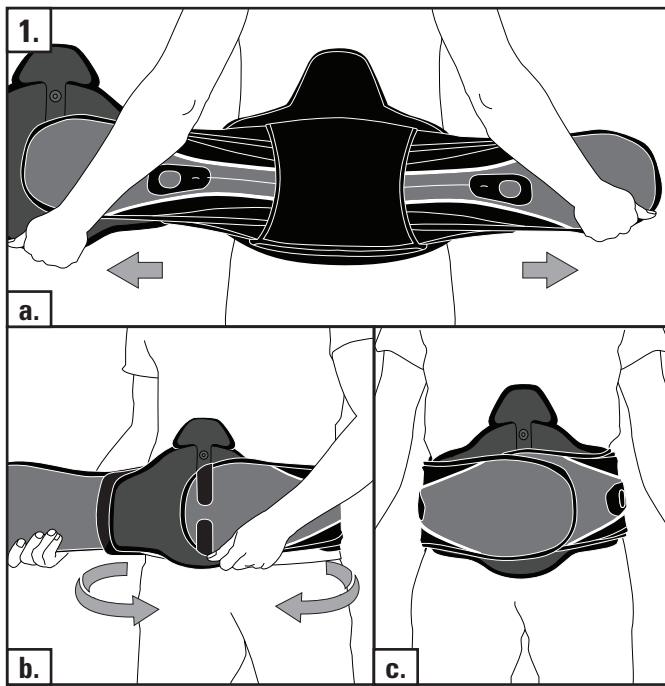ENGLISH
PLACEMENT

a. Center back panel in the middle of the back as low as is comfortable.

b. Wrap left side of belt tightly to the front of the body.

c. Repeat with right side, and while inhaling, press the hook loop material to achieve a snug fit.

Note: Be sure top and bottom of brace is snug. If needed, angle ends to create a snug fit.

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LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).
Aspen Medical Products shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.
This warranty gives you specific legal rights and you may have other rights which vary from state to state.
Warning: To ensure proper use, please review all material.
A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

PATENT PENDING
MADE IN CHINA