
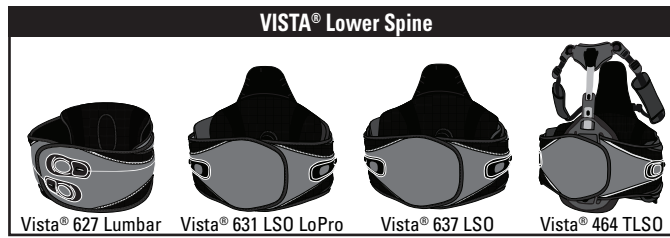
vista

LOWER SPINE

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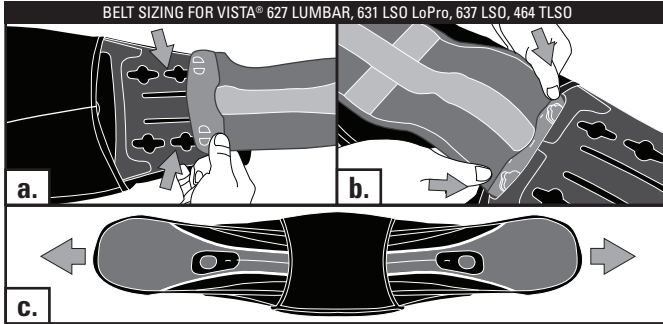Aspen Medical Products, Inc.
 6481 Oak Canyon, Irvine, CA 92618
 800-295-2776 • 949-681-0200
 www.aspenmp.com

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

* The Vista® 627 sizing follows closely to standard pants sizing. Size ranges are only guidelines. Your experiences and personal assessment could make the difference in choosing the correct size. For the Vista® 631, 637 and 464 add 2 inches for a better fit.

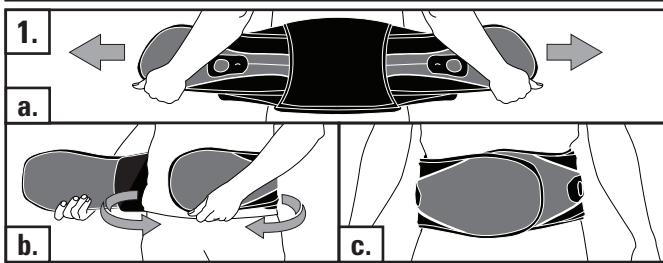
vista627 LUMBAR ORTHOSIS vista631 LUMBOSACRAL ORTHOSIS LoPro vista637 LUMBOSACRAL ORTHOSIS

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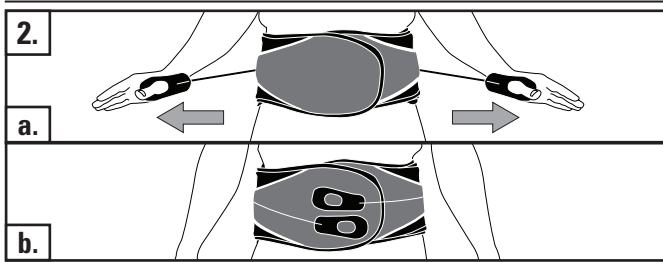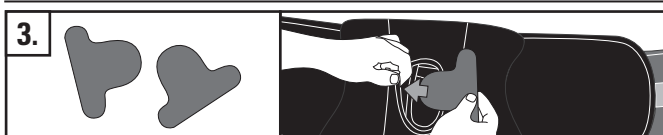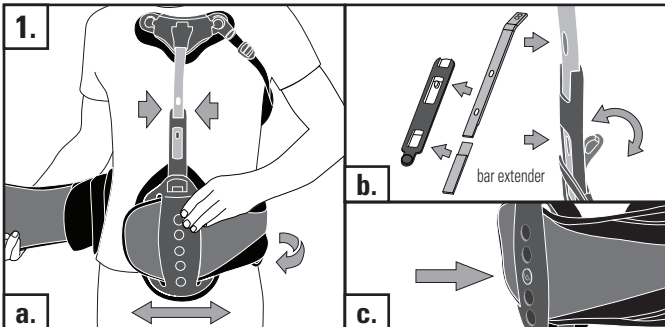
ENGLISH
PLACEMENT & ADJUST

a. Center back panel as low as possible. Slide abdominal panel and sternal pad sideways as needed to ensure brace and panel are at patient's midline.

b. For shorter patients, remove bar extender from the abdominal panel. Pry button back and slide aluminum strut so that the button locks into the appropriate hole.

c. To further adjust the sternal pad on the abdominal panel, press the button with leaf logo inward and raise or lower the sternal pad as needed.

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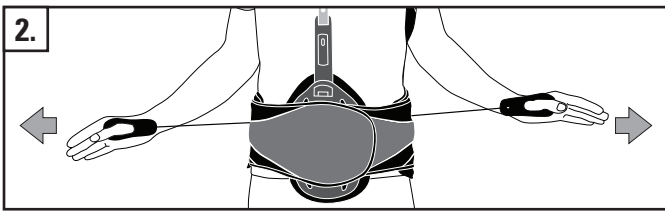
ENGLISH
TIGHTEN

Pull strongly away from body. Tighten more than you need and then loosen to achieve a proper fit. Pull left tab to tighten lower, pull right to tighten upper as needed.

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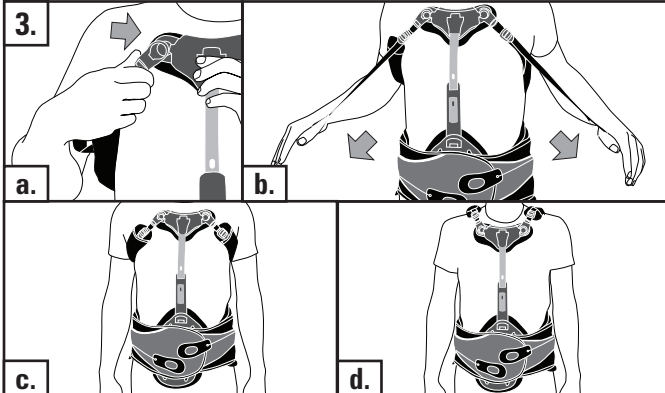
ENGLISH
STRAPS

Note: If needed, the support strut can be bent to ensure a proper fit.

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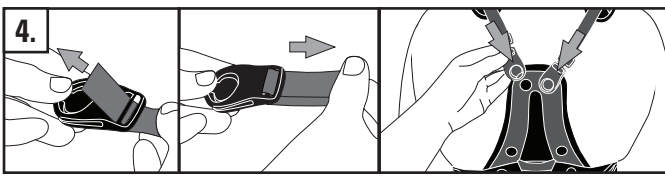
ENGLISH
OVER SHOULDER OPTION

If buckles are needed, add a buckle to strap ends and secure buckle. Attach buckles to the top of the TLSO back panel.

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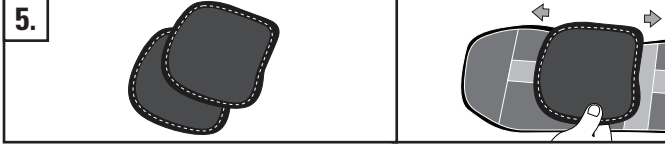
ENGLISH
ANTERIOR PANEL EXTENSION

Position anterior panel extension pads on brace for desired coverage. Panels must overlap to encircle the trunk.

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LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or parts.
Aspen Medical Products shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.
This warranty gives you specific legal rights and you may have other rights which vary from state to state.
Warning: To ensure proper use, please review all material.
A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.
THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.
PATENT PENDING
MADE IN CHINA