Indications for use

The Provox® BasePlate Adaptor (“adaptor”) is an accessory product for rehabilitation after total laryngectomy. It allows attaching medical devices with ISO 15 mm standard connector to a tracheostoma by fitting it into a Provox Adhesive base plate (“base plate”) or Provox LaryTube cannula.

A typical example would be to attach an HME with built-in oxygen adapter (TrachPhone).

For single patient use.

Contraindications

Shall not be used for mechanical ventilation.

Description of the device

This device is made according to the specifications of standard ISO 5356-1.

WARNINGS

Unconscious patients must be constantly monitored. Attached devices may inadvertently slide off the adaptor, or the base plate may detach from the tracheostoma.

Make sure the adaptor is free from oil or any other lubricant before and during use. Lubrication may lead to detachment of devices from the adaptor, or of the adaptor from the base plate.

PRECAUTIONS

Read and follow the instructions of the medical device to be used with the adaptor.

Never use force while attaching or detaching devices or the adaptor itself.

Do NOT sterilize! The device is for single patient use only.

Do not use if the device shows any signs of cracking or structural damage.

Cleaning and maintenance

Clean the Provox BasePlate adaptor daily as instructed in figure 1-2.

Disinfect the Provox BasePlate adaptor with either hydrogenperoxide 3% for 60 minutes, ethanol 70% for 10 minutes or isopropylalcohol 70% for 10 minutes at least once a day (figure 3).

If the Provox BasePlate adaptor looks dirty or have air dried in an area with risk of contamination e.g. by contact with a pet, dropping on the floor or other gross contamination, or if someone has a respiratory infection, the Provox BasePlate adaptor should be both cleaned and disinfected prior to use.

During hospitalization it is important to clean and disinfect the Provox BasePlate adaptor immediately after use and again just before use due to an increased risk of infection and device contamination. Use sterile water for rinsing, rather than drinking water.

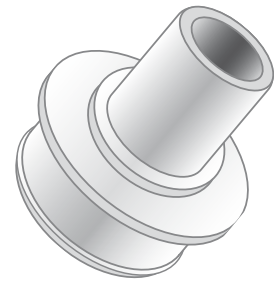
Device lifetime

When in use, the maximum life span of the device is 3 months.

Disclaimer

Atos Medical makes no warranties as to the fitness of the product for use with any third-party device and for any particular purpose.

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Provox® is a registered trademark owned by Atos Medical AB, Sweden.