NEOTECH® Products LLC Little Sucker® NeoSucker®

REF Catalog No.	DIRECTIONS FOR USE
N204	N207
N204C	N208
N204TP	N215
N205	N224
N205C	N224TP
N205TP	N225
N206	N225TP
	INSTRUCTIONS D'UTILISATION
	ISTRUZIONI PER L'USO
	INSTRUCCIONES DE USO
	INSTRUÇÕES DE UTILIZAÇÃO
	GEBRAUCHSANWEISUNG
	NÁVOD K POUŽITÍ
	ZALECENIA DOTYCZĄCE
	UŻYTKOWANIA
	ANVÄNDARINSTRUKTIONER
	KÄYTTÖOHJEET
	إرشادات الاستخدام

Manufactured by NEOTECH PRODUCTS LLC 28430 Witherspoon Parkway, Valencia, CA 91355, USA 1-800-966-0500 • neotechoroducts.com

	EMERGO EUROPE Prinsessegracht 20 2514 AP, The Hague, The Netherlands
--	--

	Medical Device
--	----------------

Made in USA

	D454 Rev K
--	------------

<i>English</i>

INDICATIONS FOR USE

The Little Sucker/NeoSucker is indicated for use as a tip for suction tubing to aspirate fluids from the mouth and nares.

The Little Sucker Nasal Tip is indicated for use as a tip for suction tubing to aspirate fluids from the nares.

The Little Sucker Cover is indicated for use to protect the Little Sucker Oral and Nasal Suction Device.

DIRECTIONS FOR USE

Step 1 If applicable, unplug thumb port and slide suction tip out of cover.

Step 2 Connect barbed end of suction tip to suction line.

Step 3 Set suction to appropriate pressure per hospital protocol or physician recommendation.

Step 4 Block thumb port with thumb to begin suctioning.

Step 5 Suction mouth or nares with intermittent suctioning.

Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.

Step 6 Suction sterile water through suction tip after each use.

Step 7 If applicable, reinsert suction tip into cover.

Step 8 If applicable, insert port plug of cover into thumb port of suction tip.

Step 9 Replace suction tip/cover every 24 hours or per hospital protocol, whichever is sooner.

CAUTIONS & WARNINGS

For oral or nasal use only.

Inserting the suction tip too far into the mouth or nares can cause tissue trauma.

The suction tip/cover should remain free of debris.

The Little Sucker Cover is designed for use with the Little Sucker Oral and Nasal Suction Device specifically. It may not accommodate other suction devices. It is not intended for use with the Little Sucker 2-piece or Little Sucker Nasal Tip.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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