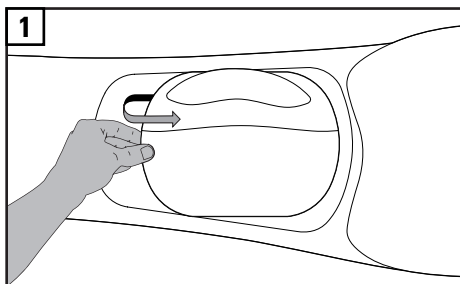
PEAK SCOLIOSIS BRACING SYSTEM

ASPEN MEDICAL PRODUCTS
 6481 Oak Canyon, Irvine, CA 92618-5202, USA
 800-295-2776 · 949-681-0200 · aspenmp.com

DUAL STRUT CONFIGURATION KIT

Note: Refer to the Peak Scoliosis Bracing System IFU for fitting instructions.

ENGLISH
REMOVE LATERAL PANEL
 Remove the lateral panel from the belt by peeling up to separate the hook and loop. Flip to expose the opposite side of lateral panel.

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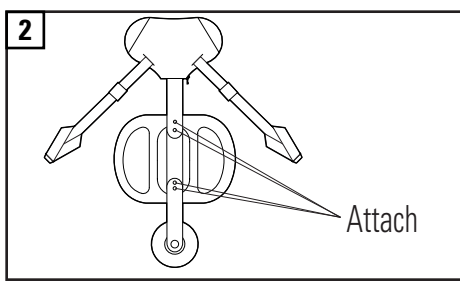
CONNECT STRUTS
 With a Phillips screwdriver, attach the thoracic strut and trochanter strut to the lateral panel using the 4 screws provided to create universal strut assembly.

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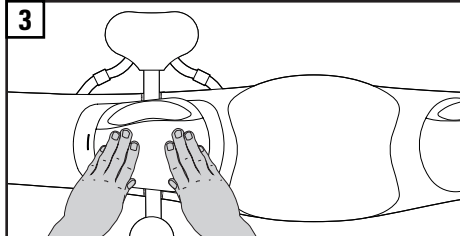
ATTACH STRUT ASSEMBLY
 Position the universal strut assembly on the belt in the desired location. Press down to secure.

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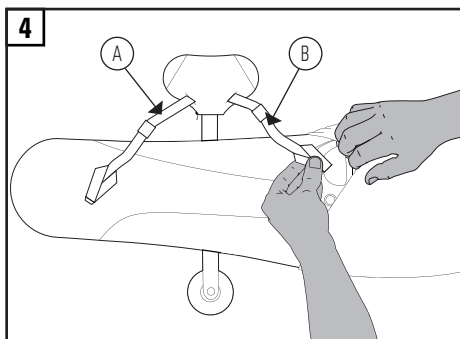
ATTACH TENSION STRAPS
 A. Attach anterior tension strap as shown.
 B. The posterior tension strap connects to the loop material on the mechanism.
 Tighten as needed once the brace has been fit to the patient.
Note: Refer to the Peak Scoliosis Bracing System Instructions for Use (IFU) for fitting instructions.

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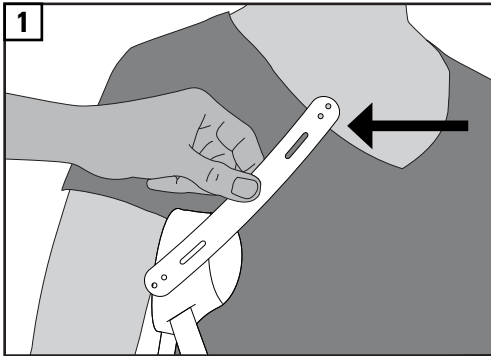
PEAK SCOLIOSIS BRACING SYSTEM

OPTIONAL CHEST PAD
Intended for kyphotic patients and patients who need de-rotation

1 IDENTIFY CORRECT SIDE

If chest pad is required, determine which end of the chest strut the chest pad will attach to.

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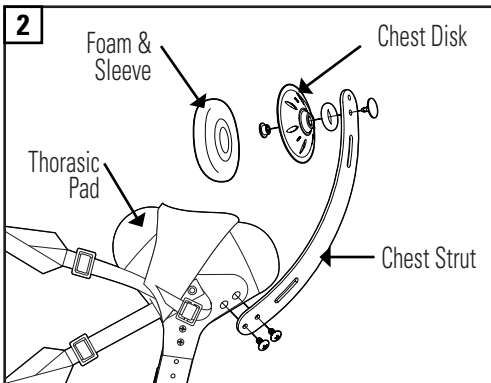
2 ATTACH CHEST PAD

Use male and female rivets to assemble the chest disk to the chest strut. Then add the foam and sleeve.

Unzip the thoracic pad sleeve to expose the two points where the chest strut will attach. Use the two screws provided to attach the chest strut. Once in place, zip the thoracic sleeve closed.

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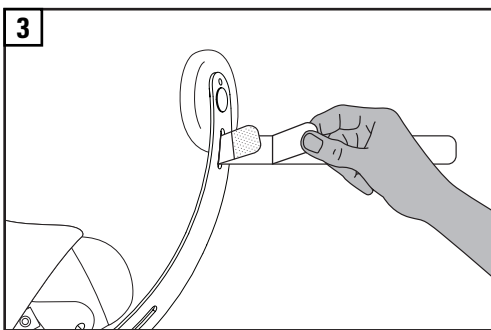
3 ADD CHEST STRAP

If patient requires left and right chest pads you can add the chest strap. Insert the loop end of strap through the slot in the chest strut and sandwich the loop end between the two hook areas.

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LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).
Aspen Medical Products shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.
This warranty gives you specific legal rights and you may have other rights which vary from state to state.
Warning: To ensure proper use, please review all material.
A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

PATENT PENDING