AIRIAL CHAMBER

HOLDING CHAMBER FOR METER DOSE INHALERS

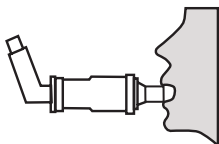
Model # MQ8000

1


Remove MDI canister mouthpiece cover. Insert MDI canister mouthpiece into the elastic opening at the back of the AIRIAL CHAMBER.

2


Insert CHAMBER mouthpiece between teeth, seal lips around mouthpiece.

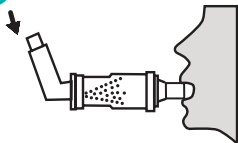3

Sit up straight and exhale. Release one puff of MDI Medication into CHAMBER.

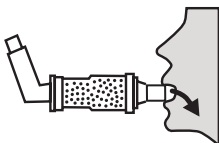4

Take a slow, deep breath through the mouthpiece. Hold breath for 5-10 seconds.

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

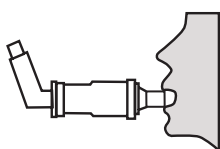
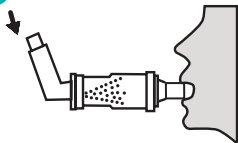
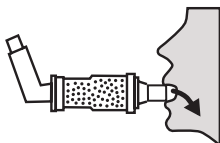
Model # MQ8001

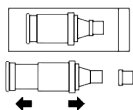1


Remove AIRIAL CHAMBER from package. Extend to full length into locked position and remove cap.

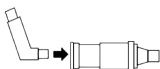2


Remove MDI canister mouthpiece cover. Insert MDI canister mouthpiece into the elastic opening at the back of the chamber.

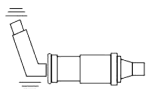3


Shake MDI canister and AIRIAL CHAMBER assembly 3-4 times.

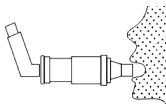4


Put Aerial Chamber mouthpiece between teeth. Close lips around mouthpiece.

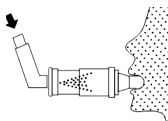5

Exhale normally. Press down on MDI canister to release medication.

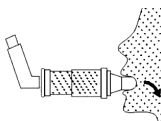6

Take a slow, deep breath through the mouthpiece. Hold breath for 5-10 seconds. Exhale through the mouthpiece. Take a second breath to ensure that all medication has been consumed.

Cleaning Instructions: Wash the Aerial Chamber in warm soapy water. Rinse with clean water and set aside to dry.

Precautions: Over time, the elastic rear opening and/or one-way valves may harden and not work effectively. Replace with a new Aerial Chamber.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

Manufactured For: Drive Medical
99 Seaview Blvd.
Port Washington, NY 11050
Tel: (866) 694-5085

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