

McKesson

Moisture Wicking Fabric

WITH ANTIMICROBIAL SILVER | SINGLE PATIENT USE

For use with MFR #s 16-1036S and 16-10144R.

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

McKesson Moisture Wicking Fabric with Silver (Ag) is a non-sterile skin protectant which is made of a stretch polyester fiber impregnated with antimicrobial metallic silver. The fabric with Silver (Ag) is designed to provide moisture transportation to keep skin dry whilst reducing odor. Moisture Wicking Fabric with Silver (Ag) acts as a lubrication aid thereby reducing skin-to-skin friction and provides a protective environment for the skin. Moisture Wicking Fabric with Silver (Ag) reduces colonization of bacteria and yeasts such as *Staphylococcus aureus*, *Staphylococcus epidermidis*, *Pseudomonas aeruginosa* and *Candida albicans* within the fabric.


INDICATIONS: McKesson Moisture Wicking Fabric with Silver (Ag) is indicated for use in the management of skin folds and other skin-on-skin contact areas.

CONTRAINDICATIONS: McKesson Moisture Wicking Fabric with Silver (Ag) should not be placed directly onto an open wound or breached skin. Do not use on individuals with known sensitivity to polyester, nylon and silver.

PRECAUTIONS

- If reddening, pain and/or irritation occurs, consult a healthcare professional or doctor.
- May not be used to protect against urine and feces. If such soiling occurs, the fabric must be changed immediately.
- Do not use the same device for multiple patients. Do not wash the fabric as it may compromise the performance of the fabric.
- Creams or ointments used in conjunction with McKesson Moisture Wicking Fabric with Silver (Ag) may reduce the efficacy of the product.
- The safe use of McKesson Moisture Wicking Fabric with Silver (Ag) during pregnancy and lactation has not been demonstrated.
- Avoid contact with electrodes or conductive gels during electronic measurements e.g. electrocardiograms (ECG) and electroencephalograms (EEG).
- The dressing must be removed prior to patients undergoing Magnetic Resonance Imaging (MRI) examinations.
- McKesson Moisture Wicking Fabric with Silver (Ag) must not be ingested and must be kept away from animals.
- Do not use product if expired. (Expiry can be found on the packaging).
- The safe use of McKesson Moisture Wicking Fabric with Silver (Ag) with secondary dressings has not been demonstrated.
- Do not sterilize the fabric.

DIRECTIONS FOR USE

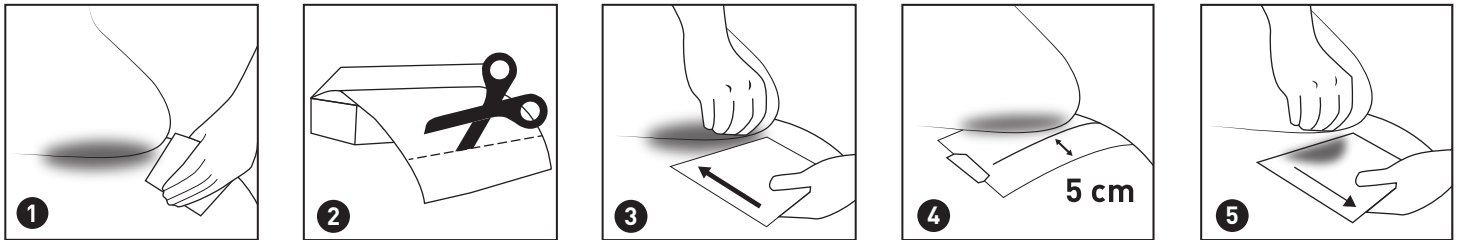

SITE PREPARATION

1. Wash skin gently. Pat dry, do not rub (Fig. 1).

DRESSING PREPARATION

1. Cut the appropriate size of Moisture Wicking Fabric with Silver (Ag) for the affected area with clean scissors, allowing a minimum of 2 inches of the fabric exposed outside the skin fold for moisture evaporation (Fig. 2).

DRESSING APPLICATION

1. Lay a single layer of fabric in the skin fold, placing one edge of the fabric in to the base of the fold (Fig. 3). Either side of the fabric can be applied in the skin fold. Gently smooth the rest of the fabric over the skin, keeping it flat. Leave at least 2 inches of fabric exposed outside the skin fold (Fig. 4).
2. Secure the fabric in one or more of the following ways: with the skin fold, with a small amount of medical tape, and/or tucking under clothing. Ensure the applied fabric is not constricting the affected area.

DRESSING CHANGE AND REMOVAL

1. Each piece of McKesson Moisture Wicking Fabric with Silver (Ag) may be used up to a maximum of 5 days, depending on fabric soiling, odor, amount of moisture and general skin condition.
2. Remove the fabric before bathing/showering and reapply when finished.
3. When removing the fabric from skin folds, gently separate the skin fold and lift the fabric away (Fig. 5).

WARNING: May cause transient skin discoloration.

CAUTION: Federal [US] Law restricts this device to sale by or on the order of a physician [or properly licensed practitioner].

Do not store above 77°F (25°C). Store in a clean, dry environment in original packaging prior to initial use and once opened. Moisture Wicking Fabric with Silver (Ag) should be stored away from harmful substances. Do not use if package is open or damaged. Do not use if perforated seal is broken or damaged. Not made with natural rubber latex. Dispose of the used dressing after a maximum of 5 days use time according to local procedures and guidelines.

Moisture Wicking Fabric with Silver is a single-patient reusable device.

Questions? Call 1-800-777-4908



Distributed by McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN A0220 OT92, rev. Draft 3 June, 2019
Made in UK

Symbol Definitions	
Symbol	Definition
R _x ONLY	Caution: Federal law (USA) restricts this device to sale by or on the order of a licensed practitioner
	Consult instructions for use
	Caution, consult accompanying documents

R_x ONLY