

Installation Directions

Models - 1875M, 1885M, 5075M & 5085M For Home Style Electric Leisure Beds

Step 1: Remove mattress. Place the assembled security rail(s) on the bed foundation, allowing at least 10" from the top of the bed. For Models 1875M and 5075M the long utility straps should be attached to the "U" tube while, the short utility straps will be attached to the security rail receiving tubes. For Models 1885M and 5085M, long utility straps will be attached to one set of receiving tubes while the short utility straps will be attached to the receiving tubes on the opposite side of the bed.

Models—1875M & 5075M



Models—1885M & 5085M



Step 2: Place the mattress back on the bed foundation. Take the long utility straps and wrap over the top of the mattress and secure to the short utility straps on the opposite side of the bed. Tighten the straps to prevent the rails from shifting.



Step 3: With the Security Rails in place you can now put the top sheet on the mattress covering the utility straps. The Security Bed Rails will now raise and lower with the electric style bed providing you a safe and secure bed environment.

