

limited 1 year warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



2017 Medical Depot, Inc. All rights reserved.
Drive DeVilbiss Healthcare
99 Seaview Boulevard
Port Washington N.Y. 11050 USA
Made in China

www.inspiredbydrive.com



otter bath chair

OT 1000, OT 2000, OT 3000



EU Authorized Representative
Drive Medical LTD
Ainleys Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.inspiredbydrive.com

recommended use

The Otter Bath Chair provides support during bathing in the bathtub. The seat, back and legs are fully adjustable. All adjustments are tool-free.

assembly

Your Otter Bath Chair comes in one carton completely assembled. This carton contains: Bath chair, head cushion strap, positioning and leg straps. If your shipment is incomplete, please call a Inspired by Drive authorized dealer.

Optional Accessories:

Tub Stand: Item # OT 8010

Shower Stand: Item # OT 8020

Soft Fabric Kit for OT 1000: Item # OT 1002

Soft Fabric Kit for OT 2000: Item # OT 2002

Soft Fabric Kit for OT 3000: Item # OT 3002

maintenance and care

1. Inspect the chair periodically for cracks, breaks, loose or missing parts or malfunctions. Check the fabric for tears or worn areas. If a product needs service or replacement parts, contact an authorized Inspired by Drive dealer.
 2. Wash chair and fabric with soap and water or 10% bleach. Do not use pine oil cleaners. Fabric and straps may be removed and machine-washed. Engage hook and loop before washing. Fabric cover may be washed on cool setting and drip-dried. Straps may be washed in warm water and dried on cool setting. Do not iron.
-

caution

Adult supervision is required at all times. The straps and supports must be carefully adjusted for the comfort and security of the child. Child must be strapped in at all times.

adjustments

See Figure 1

1. Head cushion strap height can be adjusted by opening strap in the back and positioning it at desired height. Securely engage hook and loop of strap once positioned. Slide head cushions (A) along the strap to adjust width.
2. Straps (D) can be adjusted as needed by inserting in any of the loops along the entire unit.
3. To adjust legs (E), pull crossbar (F) outward, rotate legs & push in to engage.
4. To adjust seat angle, pull on Uni-Bar under the seat (Fig 2) rotate seat to desired position and release Uni-Bar. To adjust back angle using Uni-Bar behind the Otter back (Fig 3).



figure 2

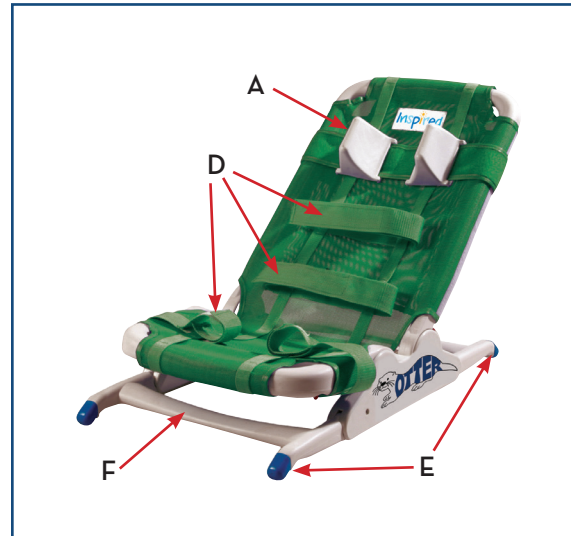


figure 1



figure 3

limited 1 year warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

2017 Medical Depot, Inc. All rights reserved.
Drive DeVilbiss Healthcare
99 Seaview Boulevard
Port Washington N.Y. 11050 USA
Made in China

www.inspiredbydrive.com

otter bath chair with soft fabric

OT 1000 SF, OT 2000 SF, OT 3000 SF



EU Authorized Representative
Drive Medical LTD
Ainleys Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.inspiredbydrive.com

recommended use

The Otter Bath Chair provides support during bathing in the bathtub. The seat, back and legs are fully adjustable. All adjustments are tool-free.

assembly

Your Otter Bath Chair comes in one carton completely assembled. This carton contains: Bath chair, head cushion strap, positioning and leg straps. If your shipment is incomplete, please call a Inspired by Drive authorized dealer.

Optional Accessories:

Tub Stand: Item # OT 8010

Shower Stand: Item # OT 8020

maintenance and care

1. Inspect the chair periodically for cracks, breaks, loose or missing parts or malfunctions. Check the fabric for tears or worn areas. If a product needs service or replacement parts, contact an authorized Inspired by Drive dealer.
2. Wash chair and fabric with soap and water or 10% bleach. Do not use pine oil cleaners. Fabric and straps may be removed and machine-washed. Engage hook and loop before washing. Fabric cover may be washed on cool setting and drip-dried. Straps may be washed in warm water and dried on cool setting. Do not iron.

caution

Adult supervision is required at all times. The straps and supports must be carefully adjusted for the comfort and security of the child. Child must be strapped in at all times.

adjustments

See Figure 1

1. Head cushion strap height can be adjusted by opening strap in the back and positioning it at desired height. Securely engage hook and loop of strap once positioned. Slide head cushions (A) along the strap to adjust width.
2. Straps (D) can be adjusted as needed by inserting in any of the loops along the entire unit.
3. To adjust legs (E), pull crossbar (F) outward, rotate legs & push in to engage.
4. To adjust seat angle, pull on Uni-Bar under the seat (Fig 2) rotate seat to desired position and release Uni-Bar. To adjust back angle using Uni-Bar behind the Otter back (Fig 3).



figure 2

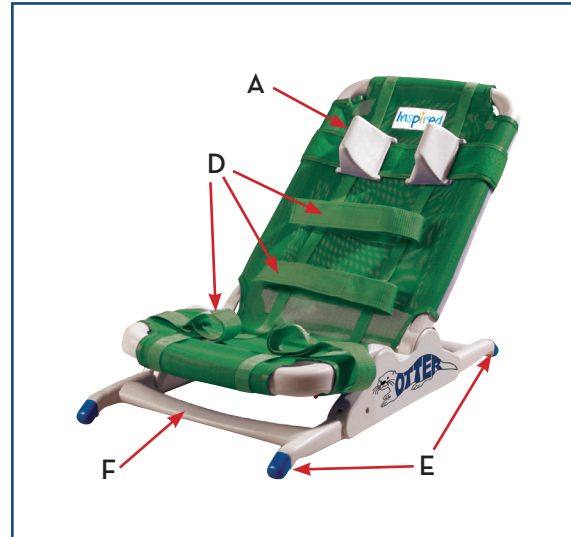


figure 1



figure 3