

3101 ERGOBELT®



- Four ergonomic, easy grip handles
- Nylon belt to assist in walking and transferring
- Durable plastic buckle with comfort finish

The ErgoSafe® ErgoBelt® combines the basic principles of ergonomics into an extremely durable and comfortable patient moving aid. With four large handles and an adjustable buckle to employ the proper fit, this transfer/gait belt is an ideal solution to implement in your health care facility or nursing home for safe patient handling.

Belt fits 22"- 49" (56 - 124.5 cm)

ERGOSAFE PRODUCTS
A PRISM MEDICAL COMPANY



3101 ERGOBELT®

USE INSTRUCTIONS:

The ErgoBelt® with its loop handles allows caregivers to assist, when needed, those patients who can walk on their own, however, may need extra attention. It is not intended for patient lifting but may be used to assist in transferring.

For proper buckling, be sure to first check that the straps are not twisted. Slide the buckle clip into the buckle lock until securely fastened. To remove the belt, simply press the sides of the buckle clip together and slide outward.

It is important to always explain to the patient your plan to transfer and how the belt will assist. Fasten the buckle to fit snugly around the patient's torso. The belt can be used around the upper body, just under the arms or around the waist. Ensure that the patient's garments are not causing discomfort where the belt is applied and be aware that clothing may ride up if the belt is not properly fitted.

Important Prior To Usage

This is only a guide, as alternative methods may be used for patient handling. We strongly recommend that your facility administrator or risk manager review and approve all protocols used for safe patient handling. Prior to using any equipment, be sure to check for signs of wear to ensure proper functionality. When using the ErgoBelt® for the first time, practice with a fellow caregiver until you have a level of comfort with the product.

LAUNDRY INSTRUCTIONS:

The ErgoBelt® Plus can be machine-washed at 80°C/170°F. Line dry for maximum life or tumble dry on low heat at 40°C/104°F

For more information on this and other ErgoSafe® products visit www.ergosafe-products.com

ERGOSAFE PRODUCTS
A PRISM MEDICAL COMPANY