


MICROCHAMBER[®]

Aerosol Holding Chamber for Use With Metered-Dose Inhalers


Before using your MICROCHAMBER medical device, please read this leaflet carefully and follow these instructions.

The MICROCHAMBER unit is a medical device that attaches to a metered-dose aerosol inhaler (MDI) prescribed by your doctor.

The purpose of the MICROCHAMBER unit is to block and disperse the large aerosol drug particles released from the metered-dose inhaler, thus allowing you to inhale the medication more effectively after spraying the MDI.

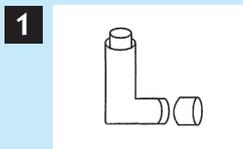


1 Remove the cap from inhaler.

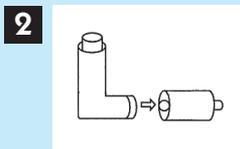


2 Insert inhaler mouthpiece into MICROCHAMBER end.




3 Shake well 4 or 5 times.

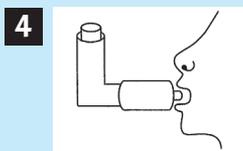


4 Place MICROCHAMBER into mouth. Close lips.

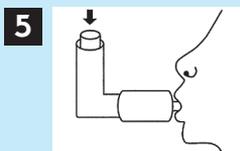

5 Spray one puff from inhaler into the MICROCHAMBER.

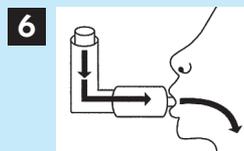

6 Breathe in slowly and fully.

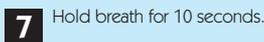

7 Hold breath for 10 seconds.



8 Repeat steps 3-7 as instructed by your doctor.

CLEANING

With repeated use, powder will accumulate on the MICROCHAMBER screen and around the MDI valve. This may eventually interfere with effective use. We suggest washing the metered-dose inhaler mouthpiece and the MICROCHAMBER unit daily or at least

once a week. Warm tap water passing through the back of the screen is best followed by drying with a lint-free cloth or leaving the MICROCHAMBER overnight to dry. Dishwasher Safe, Anti-Static Delivery, not made with natural rubber latex.

TECHNICAL INFORMATION

The MICROCHAMBER device is made of a durable polypropylene plastic material. The holding chamber and built in screen blocks large aerosol particles that normally deposit in the mouth and throat while allowing the smaller treatment particles to pass through and into the lungs. This provides effective treatment and helps to reduce unwanted side effects.

The MICROCHAMBER delivery system ensures you adequate coordination time with the spraying of your inhaler. MICROCHAMBER will help you inhale your aerosol medication more reliably. Should you have any problem using MICROCHAMBER, please contact your doctor or pharmacist.

Caution: Federal law restricts this device to sale by or on the order of a physician or other licensed practitioner.

Marketed by Respiratory Delivery Systems, Inc.
6 Courthouse Lane, Suite 8, Chelmsford, MA 01824

© Respiratory Delivery Systems, Inc. 2015 Rev.2
www.rdsusa.com