


**Fabrifoam®
CarpalGard™ &
Ultra CarpalGard™
With CMC Support**

An innovative version of the CarpalGard™ with added support for the CMC joint. This 2-part device allows for customized individual fit to maximize thumb and wrist support and positioning.




**Fabrifoam®
Ultra CarpalGard™
with CMC Support**

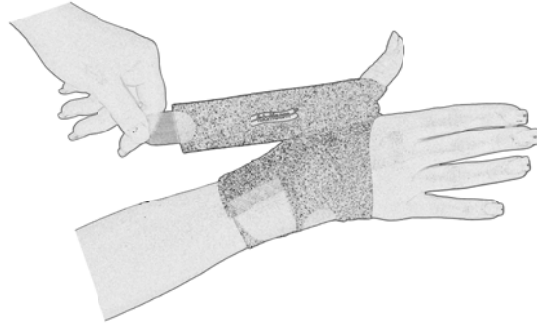

Step 1
Place thumb through slot (foam side to skin) with short strap positioned on the back of hand.

Step 2
Wrap long strap snugly around and over short, and secure with internal Velcro® dot.

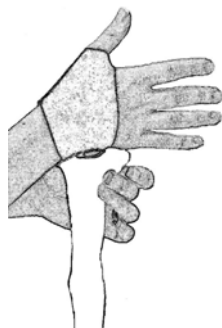


Step 3
Continue wrapping strap snugly around hand and wrist area and secure with Velcro tab.



On Ultra Version:

Step 4 - Customized support.
Additional strap is included to customize support for specific ailment or injury.

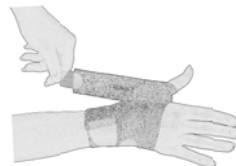

Attach one end of the strap to the device using Velcro tab. Wrap strap around thumb to reposition, compress, support or immobilize the CMC joint area as needed. Secure with Velcro tab.

This strap may also be wrapped around wrist for additional wrist support or stabilization as required.

(You should feel comfortable support – do not over-tighten. Re-wrap to adjust compression if necessary.)