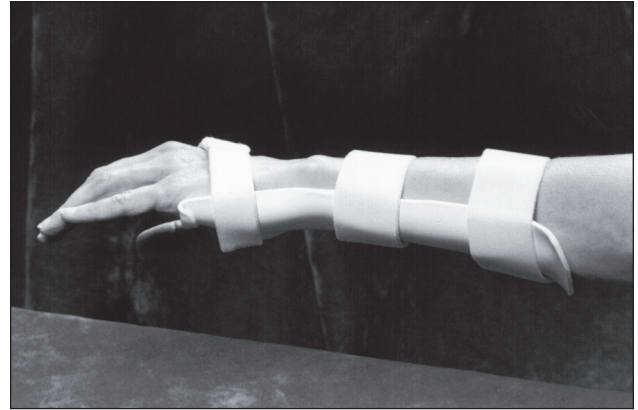
## Rolyan® Radial Bar Wrist Cock-Up Splint:

A171-1P, A171-1A1, A171-220; A172-1P, A172-1A1, A172-320; 5504-46, 5504-47; 5514-0001, 5514-28, 5514-30; A921-730, A921-731, 5664-03 through 5664-05

A

B

C

D

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints listed.

### INDICATIONS

- To immobilize the wrist
- May be used for arthritis, carpal tunnel syndrome, sprains, and fractures

### CONTRAINDICATIONS

- Not for the acute phase of most wrist fractures
- Not in the presence of active MCP synovitis
- Not in the presence of severe edema

### PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

### CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water and allowed to dry thoroughly before reapplication.

### PROCEDURE

The splint blank can be applied to either hand—simply invert the splint blank.

1. Heat splint blank as follows:

Soften in water heated to approximately 160°F (70°C) until pliable. (Refer to the instructions for Rolyan splinting materials for information about working with specific splinting materials.) Remove from heat pan and place on a towel on a flat surface. Blot excess water.

**Fabricating a volar splint** (see instructions 8–12 for a dorsal splint)

2. If possible, place patient's hand in supination to achieve a gravity-assisted position.
3. Standing in front of the patient, place the splint with the radial bar through the web (A).
4. Fold over the distal edge to clear the distal palmar crease and flare or trim the splint to clear the thenar eminence.
5. Mold the radial bar around the hand and mold the splint symmetrically on the forearm (A).

6. The splint can be trimmed and the edges flared if necessary.

7. Apply appropriate straps. A completed volarly applied splint is shown in B.

### Fabricating a dorsal splint

8. If possible, place patient's hand in pronation to achieve a gravity-assisted position.
9. Standing in front of the patient, place the distal end of the splint over the dorsum, proximally to the MCP heads (C).
10. Mold the radial bar around the hand and mold the splint symmetrically on the forearm (C).
11. The splint can be trimmed and the edges flared if necessary.
12. Apply appropriate straps. A completed dorsally applied splint is shown in D.

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**Patterson Medical, Ltd.**