

Abduction Wedge

Reorder #	Unit
703075	Each

Purpose

Designed to prevent the leg adduction that can result from extended periods of sitting with knees held together. Recommended for wheelchair resident/patients with spinal cord or neurological injuries. Maintains four-inch knee separation.



Easy To Use

1. Place wedge between resident's/ patient's thighs as show in illustration at top of sheet.
2. Place straps under and around thighs. See illustration A.
3. One strap has a strip of Velcro[®] hook material sewn on it. Secure this strap - Velcro[®] strip facing up - to the top of the wedge. Strap should fit snugly around thigh. See illustration B.
4. Place opposite strap over strap with Velcro[®] strip and secure. Strap should fit snugly around thigh. See illustration C.



Easy to Care For

Nylon wedge cover cleans easily with spray-type cleanser and a damp cloth. Do not use a cleanser that contains bleach. Lint and other particles may accumulate in Velcro[®] hook material. These may be remove using a stiff brush.