
Vibrating Light #458

USER'S GUIDE



Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com

Now with bright LED!

We've updated our best selling vibrating light with the latest technology in illumination. This fun light encourages individuals to reach out. The textured plate light up and vibrates when pressed, increasing visual, auditory and tactile experiences. Also available with music, lights and vibration. Both versions can be used as a switch. Can be placed on a lapboard or mounted on a wall (mounting hardware included). Size: 9" L x 2" H x 12"W. Requires: 4 AA Batteries.

Operation:

1. The Vibrating Light requires 4AA batteries (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartments are located on the back of the unit. To remove the battery compartment covers, unscrew the small Phillips Head screws, and then lift the covers off. Install new batteries, being careful to observe proper (+) & (-) battery polarity. Replace covers and screws. Turn the On/Off switch to On.
3. Pressing on the textured plate will illuminate the plate along with vibration for as long as the plate is depressed.
4. The Vibrating Light can also be used as a switch by connecting one end of the included 1/8" to 1/8" male cord into the jack on the Vibrating Light, and the other end into your switch adapted toy or device. The toy/device will activate as long as the textured plate is depressed.

Please Note: The vibration and light will also activate while your toy/device is being activated.

Troubleshooting:

Problem: The Vibrating Light does not work.

Action #1: Check all batteries for proper (+) & (-) battery polarity.

Action #2: Replace the batteries with fresh new batteries. Never mix different brands or old and new batteries.

Care of Unit:

The Vibrating Light can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

When finished playing with the Vibrating Light turn the ON/OFF switch to OFF to help prevent battery drain. For prolonged storage we recommend removing the AA batteries.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.