

Amoena Breast Form Care Instructions

Amoena breast forms are incredibly resilient and are designed to stand up to the normal everyday demands placed upon them. If you take care of your breast forms properly, they may be retained for many years.

However, take care of your breast form from sharp objects such as pins or pet claws as it's not possible to repair it.

Use of talcum powder and body lotions on or around your breast form should also be avoided.

Cleaning of Your Silicone Breast Prosthesis

If you want your breast form in good condition, wash it everyday with soap and water and don't forget to dry it with a soft cloth or towel.

When you are not wearing it, just make sure that you keep your breast form in the cradle provided. This will help to maintain its shape.

Amoena Contact Adhesive Breast Form Care Instructions



1. Put three drops of the Soft Cleanser on the dry adhesive surface and then moisten the special brush with lukewarm water.



2. Now scrub the adhesive surface with the special soft brush firmly for about three minutes in circular movements.



3. Then rinse with plenty of lukewarm water.



4. Finally, allow to air-dry.