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Protocollo per l'uso del DeClogger®

Scopo

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
Rx ONLY

MADE IN
THE U.S.A.

The following Bionix DeClogger's are compatible with ENFit connection. #911-Blue, #912-Yellow, #921-Brown, #922- Orange. ENFit® and NRFit™ are trademarks of GEDSA

©2019 Bionix Development Corp. | 5154 Enterprise Blvd., Toledo, Ohio 43612
Phone: 800.551.7096 | Fax: 800.455.5678 | Email: bionix@bionix.com
RM95-0007 | Rev. C
bionix.com

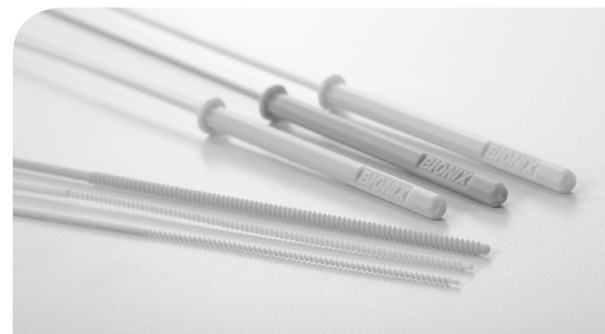DECLOGGER® - USER INSTRUCTIONS
DECLOGGER® - GEBRAUCHSANWEISUNG
DECLOGGER® - INSTRUCCIONES DE USO
DECLOGGER® - MODE D'EMPLOI
DECLOGGER® - ISTRUZIONI PER L'USO
DECLOGGER® - INSTRUÇÕES PARA O UTILIZADOR

REORDER NUMBERS:

#911 - BLUE 14-16 FR./39.5CM
#912 - YELLOW 16-22 FR./39.5CM
#913 - GREEN 20-22 FR./39.5CM
#921 - BROWN 14-16 FR./21.5CM
#922 - ORANGE 18-24 FR./21.5CM
#935 - VARIETY PACK (1 OF EACH STYLE)

DECLOGGER® - USER INSTRUCTIONS

Protocol for use of the DeClogger®

Purpose

To maintain or achieve patency of gastric and/or jejunostomy enteral tubes.

Policy

The DeClogger should be used every week to maintain unimpeded flow of an enteral formula. The DeClogger will also be used to achieve patency of a tube that has become clogged with semi-solid formula. The DeClogger should be disposed of after a single use.

Protocol

The following protocol should be followed according to the desired need.

INSTRUCTIONS

Blocked Jejunostomy Tube

1. Determine the size of the gastric or jejunostomy tube. CAUTION: Verify that the tube has not been modified, i.e. shortened.
2. Select appropriate size DeClogger that corresponds to the size of the tube.
3. Turn enteral feeding pump to PAUSE position and disconnect from enteral tube. (If gravity pump, clamp the tubing.)
4. Insert the DeClogger to reach the blockage and slowly rotate two times in a clockwise direction then reverse and rotate two times in a counter-clockwise direction while removing it. Do not attempt to force the DeClogger through the entire blockage.
5. Repeat step 4 until the stop disk of the DeClogger is reached without difficulty.
6. Flush tube with 30-60 cc's of water.
7. Once clear, reconnect the delivery tube and restart the enteral feeding.
8. Discard DeClogger and document results.

Prophylactic Use

For prophylactic use, implement the DeClogger protocol every week depending on the patient's feeding and medication scheduling.

CAUTION

If the DeClogger becomes imbedded in the blockage and cannot be easily dislodged, the enteral feeding tube will need to be changed. DO NOT attempt to physically remove the DeClogger.

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