# Brava®

# Coloplast

Skin Barrier Wipe  
Sting Free

Paño Protector de Piel  
Sin Agujón

## Instructions for use Instrucciones de uso

23315575 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S.  
All rights reserved.© (2014-04-30)

Manufacturer: Coloplast A/S  
Holtedam 1, 3050 Humlebaek, Denmark  
Distributed by: Coloplast Corp., Minneapolis, MN, 55411,  
USA  
Fabricante: Coloplast A/S Holtedam 1 3050 Humlebaek  
Denmark  
Distribuido por: Coloplast Corp., Minneapolis, MN, 55411,  
USA

## Indication

The product is intended for protection of intact skin from the damaging effects of body waste and adhesives.

## Information

The product is silicone based. The product dries within seconds and does not have an effect on the adhesion of further appliances.

Ingredients: Contains a mixture of alkyl siloxanes.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

## Caution

Avoid direct use on wound beds, eyes and mucous membranes.  
Keep away from naked flame.

## Warning

Reusing this single-use product may create a potential harm to the user. Reprocessing, washing, disinfection, and sterilization may compromise product characteristics, causing additional risk of physical harm to or infection of the user.

## How to use

1. Ensure that the skin is clean and dry.
2. Tear sachet to open and take out the tissue and unfold it.
3. Gently apply the product onto the areas to be protected.
4. Once the skin is completely dry, the appliance can be applied as usual.

A new application of the product is required at every appliance change.

The product is intended for single use only and should be disposed of with normal household waste. Do not flush the product down the toilet.

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Fabricante: Coloplast A/S Holstedvej 1