Brava®

Coloplast

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Instructions for use Instrucciones de uso

23314127 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S.

All rights reserved. © 2015-02-10

Distributed by: Coloplast Corp., 1601 W. River Rd. N,
Minneapolis, MN 55411 USA

For customer service or to return product,
please call (800) 258 3476 or (612) 337 7800

Use-by date

Batch code

Date of manufacture

Consult instructions for use

Catalog number

Keep away from sunlight.

Recyclable packaging

Indication

The Powder is intended for moisture absorption on skin around the stoma.

Information

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Warning

Brava Powder is a nonsterile product and should not be used by people with severely damaged or impaired immune systems such as people with leukemia or people taking medication to suppress their immune system after organ transplant. The Powder should not be used on premature babies, or infants receiving incubator care.

How to use

Preparation

1. Wash and dry the skin around the stoma.

Application

2. Unscrew the cap and aim at the intended area.
3. Shake or gently squeeze the bottle and spread the powder on the intended skin
4. Spread the powder evenly using using toilet paper, a finger, or gauze.
5. Dust off excess powder before applying the ostomy barrier.
6. Close the cap tightly after use and store in a cool, dry place.

Disposal

7. The product should be disposed of with normal household waste.

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