Coloplast®

Paste

 Coloplast A/S
3050 Denmark
IFU11921 The Coloplast logo is a registered trademark of Coloplast A/S.
© [2010-10.] All rights reserved Coloplast A/S, 3050 Humlebæk, Denmark.

Warning

Coloplast paste is a non-sterile product and should not be used by people with severely damaged or impaired immune systems such as people with leukaemia, or people taking medication to suppress their immune system after organ transplant. Coloplast paste should not be used on premature babies or infants who are receiving incubator care.

Caution

For external use only.

Coloplast paste contains alcohol and should not be applied directly onto an open wound or skin ulcer. When applied to irritated or abraded skin a short stinging sensation may be experienced. Consult a healthcare professional if skin irritation occurs and persists.

Coloplast accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.


Information

Coloplast Paste for people with a stoma provides extra security between the skin and the appliance. The paste can level out scars and wrinkles around the stoma.

This is a re-usable device.

Store away from direct sunlight, in a cool and dry place.

How to use Coloplast Paste

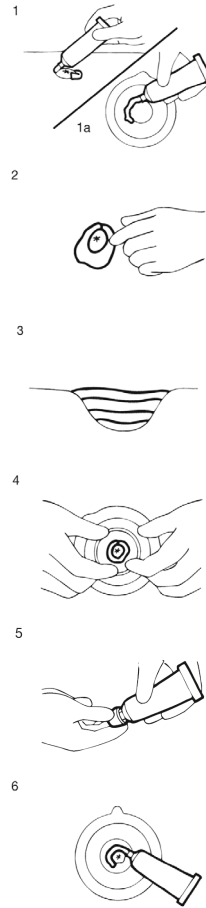Application

1. Coloplast paste may be applied directly from the tube onto the skin by forming a ring of paste around the stoma.

1a. Alternatively, it may be applied directly to the skin barrier in a ring around the opening on the skin barrier which has been cut to fit the stoma.

2. If needed, level the paste by gently smoothing using a moistened finger or spatula.

3. With deep depressions it is advisable to layer the paste allowing 30 seconds between each application.

4. Whichever method is chosen, after applying the barrier seal secure it around the stoma by pressing with a finger.

5. Recap the tube tightly immediately after use in order to prevent the paste drying and hardening.

For 2-piece users only Re-sealing between bag changes.

6. It is possible to re-seal around the stoma by placing a ring of paste close to the stoma. Level out the ring slightly to secure a proper seal. Then apply a new bag.

Removal

The paste is easily removed and will peel away with the skin barrier. A residue of paste can be left on the skin to apply a new bag or can be removed with lukewarm water and a wash cloth.

It is better to leave a slight residue than to irritate the skin by scrubbing off all the residue.

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Distribuido por Coloplast Corp