

# one year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This One Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

**drive**  
WWW.DRIVEMEDICAL.COM

*Wenzel<sup>ite</sup>Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

seat2go

item # CRS 2000 & CRS 3000



EU Authorized Representative  
Drive International, LLC  
Globe Lane - Stockport  
DUKINFIELD SK16 4RE, GB

*Wenzel<sup>ite</sup>Re/hab*

**drive**  
WWW.DRIVEMEDICAL.COM

---

## assembly

---

Your Seat2Go comes in one carton completely assembled.

---

## use & adjustments

---


Your Seat2Go can be used as a floor sitter, booster chair and as an insert in a stroller.

---

## floor sitter

---

1. Place Seat2Go on a flat surface.
2. Open Fastex™ clip (A) on pelvic belt. Detach two shoulder straps (B) from Velcro™ strips on Seat2Go back. Open Fastex™ clip (C) on chest support (D) and pull support apart.
3. Place child in Seat2Go and fasten pelvic belt.
4. Wrap chest support (D) snugly around child, attach Velcro™ and fasten Fastex™ clip.
5. Determine correct fit for shoulder straps (B) and attach to Velcro™ strips on Seat2Go back.

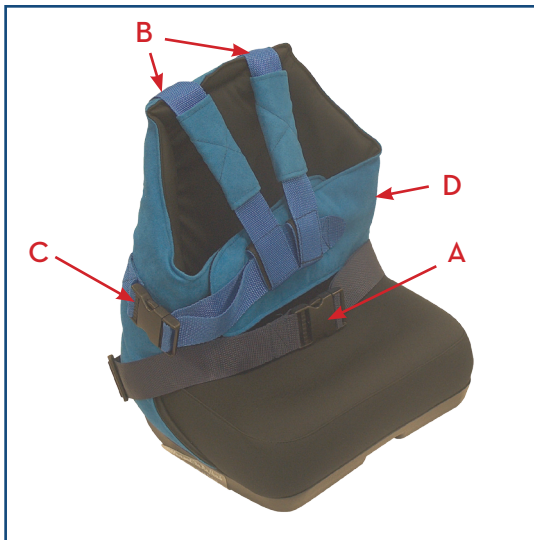


---

## booster chair and stroller

---



1. Remove fastening belts from pocket on seat bottom (E) and Seat2Go back (F).

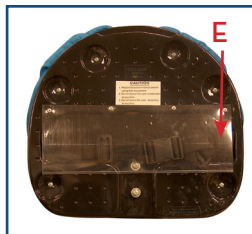

2. Place Seat2Go on any chair or in a stroller, wrap belts around the back and seat of chair/stroller and close the buckles. NOTE: Make sure belts are taut.
3. Insert child as per instructions for floor sitter.

---

## accessories

---

(sold separately)

CRS 8000  
CRS 8025

Headrest  
Abductor