


## VIPER JR

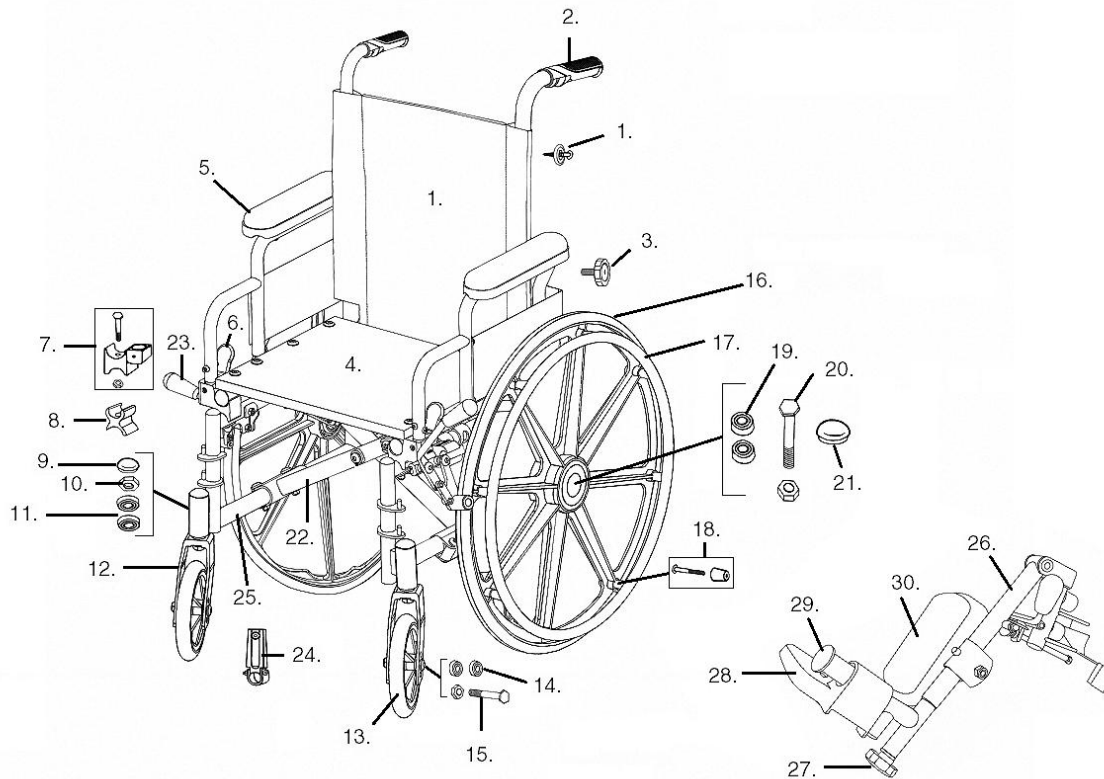

- |  |  |
|--|--|
| <p><b>1. Back Upholstery 12''- STDS2353BLK</b><br/> <b>Back Upholstery 14''- STDS2355BLK</b><br/> <b>Back Uphol. Hardware (8 ea)- STDS5D5504</b></p> <p><b>2. Hand Grip- TRHG</b></p> <p><b>3. Back Height Adjustment Knob- STDS895</b></p> <p><b>4. Seat Upholstery 12''- STDS2352BLK</b><br/> <b>Seat Upholstery 14''- STDS2354BLK</b><br/> <b>Seat Upholstery Hardware (8 ea)- STDS1072-8</b></p> <p><b>5. Armrest Left- STDSDDAJVL</b><br/> <b>Armrest Right- STDSDDAJVR</b><br/> <b>Arm Pad- STDSDSKNB</b><br/> <b>Arm Pad Screw (ea)- STDS002001</b></p> <p><b>6. Arm Release Lever- STDS4012-3</b></p> <p><b>7. Front Seat Saddle Left- STDS2A3140</b><br/> <b>Front Seat Saddle Right- STDS2A3141</b></p> <p><b>8. Rear Seat Saddle- STDS4058</b></p> <p><b>9. Dust Cap- STDS1051</b></p> <p><b>10. Fork Stem Nut- STDS2A3401</b></p> <p><b>11. Fork Stem Bearing (ea)- STDS2A3402</b></p> <p><b>12. Fork- STDS2A3403</b></p> <p><b>13. Caster- STDS3021</b></p> <p><b>14. Caster Bearing (ea)- STDS1011</b></p> | <p><b>15. Caster Axle- STDS2A3423</b></p> <p><b>16. Rear Wheel Assembly- STDS1025</b></p> <p><b>17. Hand Rim- STDS2A3413</b></p> <p><b>18. Hand Rim Hardware (8 ea)- STDS1073</b></p> <p><b>19. Rear Wheel Bearing (ea)- STDS1012</b></p> <p><b>20. Rear Axle- STDS2A3411</b></p> <p><b>21. Hub Cap- STDS1000C</b></p> <p><b>22. Cross Frame 12''- STDS2A3412</b><br/> <b>Cross Frame 14''- No Longer Available</b></p> <p><b>23. Wheel Lock Left- STDS2A340L</b><br/> <b>Wheel Lock Right- STDS2A340R</b></p> <p><b>24. Cross Frame Assist 12''- STDS2A3424</b><br/> <b>Cross Frame Assist 14''- STDS2A3422</b></p> <p><b>25. Side Frame Left- STDS2A341L</b><br/> <b>Side Frame Right- STDS2A341R</b></p> <p><b>26. Elevating Leg Rest- JLELR-TF</b></p> <p><b>27. Adj. Knob &amp; Bolt Assembly- JLELR-TFKA</b></p> <p><b>28. Footplate- STDS003441</b></p> <p><b>29. Footplate Bumper- STDS1026</b></p> <p><b>30. Calf Pad- STDS2A3442</b></p> |
|--|--|