

IndeeLift®

The Human Floor Lift

For Home & Business

In the home, IndeeLift's HFL's lift individuals from the floor after a fall or when getting back up from exercise, therapy, gardening or playing with a family pet.

**The People Picker Upper!
HFL-300/HFL-400**

For businesses, (assisted living, senior housing, senior centers, hospice and mobile caregivers), they allow caregivers to easily & safely recover fallen residents.



HFL-300-D/HFL-400-D

Self-Operation

After a fall, the fallen person scoots to the HFL, slides or tilts onto the beveled seat, pushes a button on the wired remote, activating the motor which raises them from the floor. When ready, they simply stand up and walk away.



Assisted Operation

A helper wheels the HFL to the fallen person and assists them onto the seat, if necessary. The helper pushes a button on the wired remote which raises the fallen person from the floor. The helper can either assist them up from the seat or directly transfer them to a wheelchair.





For Home & Business

Self or Assisted Operation

Homes
 Adult Communities
 Senior Centers
 Assisted Living
 Mobile Caregivers

The People Picker Upper! HFL-300/HFL-400

Description

The HFL-300 and HFL-400 are identical except for rated capacity. The HFL-300 lifts a person weighing up to 300 pounds, (136kg). The HFL-400 lifts a person weighing up to 400 pounds, (181kg).

They are also available as Disability units, (HFL-300-D and HFL-400-D), which are identical to the HFL-300/HFL-400 with the inclusion of waist and chest safety belt assemblies for additional safety when lifting individuals with significant mobility challenges.

IndeeLift's patented HFLs lift a fallen person from the floor. After a fall, a person can be easily raised without the risk of injury to themselves or any helper. In the home, the HFLs allow for fall recovery without outside assistance, allowing mobility challenged individuals to remain in their homes longer, thereby maintaining their **privacy, dignity and independence!**


Summary of Features

- Self or assisted operation
- Lifts a person weighing up to 300/400lbs. to a height of 21"/54 cm above the floor
- Beveled seat-front for easy slide-on access
- Wired remote for convenient operation
- Swinging/removable Rise Assist Handles for easy transfer to wheelchair
- Lightweight – 63/65 lbs. 29/30 kgs
- Small footprint for tight spaces
- Powder coated for easy cleaning
- Self-contained rechargeable Lithium Ion battery and battery charger

Maintain Privacy, Dignity and Independence

These products are covered by one or more patents, including U.S. Patent No. 9,808,388 B2

Major Components

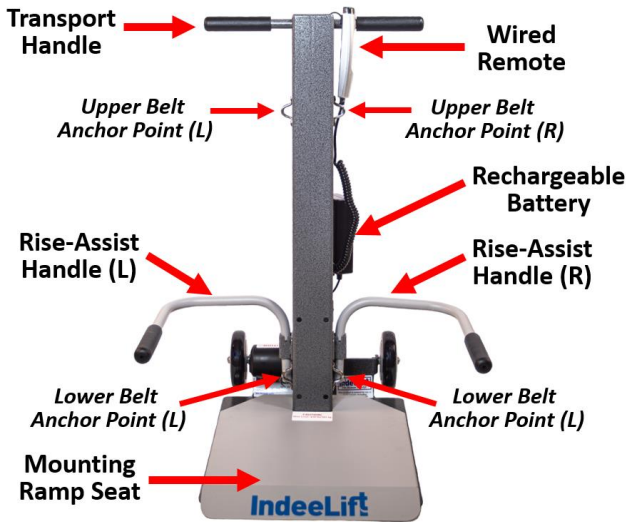

* Belt Anchor Points and Safety Belts are included with the Disability units, (HFL-300-D/HFL-400-D)

Physical Specifications

	HFL-300	HFL-400
Safe Working Load	300 lbs / 140 kgs	400 lbs / 180 kgs
Maximum Length	21 in / 54 cm	21 in / 54 cm
Maximum Height (Low)	34 in / 86 cm	34 in / 86 cm
Maximum Height (High)	54 in / 137 cm	54 in / 137 cm
External Width	15 in / 38 cm	15 in / 38 cm
Max Seat Height	21 in / 54 cm	21 in / 54 cm
Wheels	5 in / 13 cm	5 in / 13 cm
Total Weight	63 lbs / 29 kg	65 lbs / 30 kg

Electrical Specifications

Standard 3-prong AC power cord
 Operating Environment: >+5°C <+40°C
 Lithium Ion Battery: 29.9V DC 2600mAh