


THE SELF-ADHESIVE FABRIC TAPE

Mefix is the ideal fixation for dressings, catheters, and cannulae.

Nonwoven Polyester Fabric Backing

- Soft and elastic
- Non-absorbent

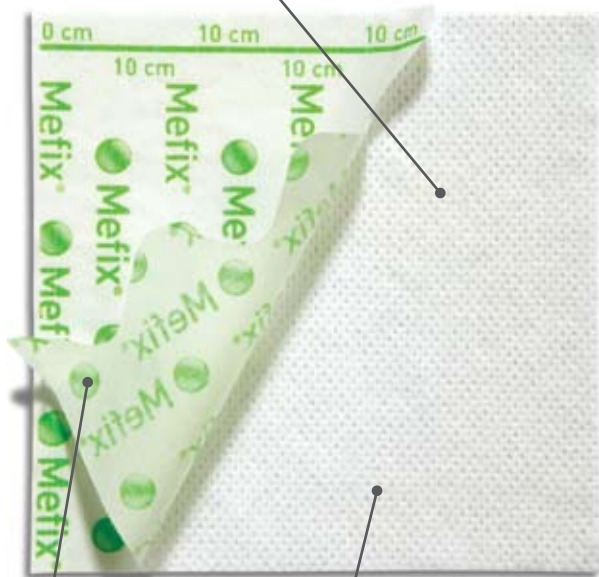

Protection Paper

- Easy-to-remove
- Pre-measured
- Perforated split

Polyacrylate Adhesive

- Skin friendly
- Water based
- Solvent free

BENEFITS OF MEFIX

- Skin friendly adhesive
- Provides secure fixation
- Stretchable, conformable fabric backing provides barrier to blood and fluid
- Pre-measured release paper allows for convenient measurement/cutting
- May be applied over flexing areas such as neck, elbows, knees
- Air permeable
- Easy-to-remove backing paper with perforated split



Please note new product codes on reverse side.



HOW MEFIX® WORKS

The porous structure of the nonwoven fabric ensures air and water vapor permeability. **Mefix** is stretchable, conforms well to body contours and allows body movements when in place. The fabric is non-absorbent and provides a barrier to blood and fluid. The polyacrylate adhesive has been designed to be gentle to the skin but at the same time provide secure fixation. The easy-to-remove protection paper is pre-measured in 4 inch intervals and has a perforated split for easy removal and application. **Mefix** may be easily cut to the desired shape and size.

AREAS OF USE

- Fixation for dressings, catheters and cannulae.

NOTE: Care should be taken to ensure that **Mefix** is not applied under tension, to prevent shearing forces causing damage to the skin. This is particularly important when applied over joints.

HOW TO USE MEFIX



Cut to appropriate size.



The two parts of the protection paper are separated. The narrow part is removed.



The part with exposed adhesive is applied to the skin. Remaining protection paper is removed. Firmly smooth **Mefix** to obtain proper adhesion.

MEFIX ORDERING INFORMATION

Packaged in boxes containing individual, non-sterile rolls.

CURRENT Product Code	NEW Product Code	Product Size	Rolls/box	CURRENT Rolls/case	NEW Rolls/case	HCPCS
310250†	310299†	1" x 11 yds (2.5 cm x 10 m)	1	39	40	A4450*
310500	310599	2" x 11 yds (5 cm x 10 m)	1	20	40	A4450*
311000	311099	4" x 11 yds (10 cm x 10 m)	1	20	20	A4450*
311500	311599	6" x 11 yds (15 cm x 10 m)	1	20	22	A4450*
312000	NA	8" x 11 yds (20 cm x 10 m)	1	10	10	A4450*

Transition to NEW codes starts in December 2009 and roll into 2010



MÖLNLYCKE®
HEALTH CARE

† This article does not have a perforated split in the protection paper

* A4450 (per 18sq. inch)

5550 Peachtree Parkway, Suite 500, Norcross, GA 30092 | 800.882.4582 | www.molnlycke.us | www.mhcwoundcare.com

The Mölnlycke Health Care and Mefix names and respective logos are registered trademarks of Mölnlycke Health Care AB. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2009. Mölnlycke Health Care AB. 1.800.882.4582.

US011280907 - 200012 10.09