

---

---

## Musical Light Box #200

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### Now with a brighter LED!

This multi-purpose musical light box can be used as a bright backlight for tracing or as a light source for people with visual impairments. Increases visual attention and is great for working on pre-writing and writing skills. Easily activated by any of our capability switches. Size: 12"L x 11"W x 4"H. Requires Capability Switch and 2 C Batteries. Weight: 1 lb.

#### Operation:

1. The Musical Light Box requires two-D size batteries. Alkaline batteries are required (Duracell or Energizer brands). Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. The battery holder is located under the unit. When installing new batteries, always be careful to observe proper battery polarity.
3. Connect the unit to any external capability switch via the jack, located on the back of the unit. The Musical Light Box #200 is equipped with a 1/8-inch jack, for use with capability switches with a 1/8-inch plug.
4. Make sure that the connections between the external capability switch and the adapter AND the adapter and the jack on the Musical Light Box are tight. There should be no gaps.
5. Activate the external capability switch to light up the box and begin the music.
6. The light and music will only remain on while the external switch continues to be activated. Once you release the switch, the light and music will go off.

#### Troubleshooting:

**Problem:** The lights and/or music do not turn on, or are not operating at full strength.

**Action #1:** Replace the batteries, paying close attention to proper battery polarity.

**Action #2:** Make sure that the connections between the switch and the adapter AND the adapter and the jack are tight.

**Problem:** A light bulb burns out.

**Action:** Remove the large, rectangular, opaque cover by gently pulling up on it to release the Velcro hold. Unscrew the light bulb and replace it with a 2.47-volt bulb. Replace the cover.

**Note:** If you cannot purchase this bulb from your local hardware store, call our Technical Service Department at 1-800-832-8697 for a replacement.

#### Care of Unit:

The Musical Light Box can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.