

McKESSON

PREVENT[®] HP

Safety Lancets, Push-Button

McKesson Prevent[®] HP Safety Lancets are push-button activated, feature the HP-Haemolance[®] Plus design, and were created with OSHA sharps safety and infection prevention guidelines in mind. The easy-to-handle design and ergonomic shape provide a firm, non-slip grip allowing easy twist-off removal of the protective cap. To activate, simply place the lancet firmly against the desired puncture site and press the push-button. Advanced safety-engineering and features—such as automatic lock-out after use—ensure the user never sees or handles the sharp. Rapid needle retraction, as well as a tri-beveled needle design, helps ensure accurate blood sampling. Available in a range of sizes to meet a variety of capillary testing needs.

FEATURES AND BENEFITS

- Automatic needle retraction and lock out ensure sharp edge is enclosed before and after use
- Convenient, intuitive, one-step activation for precise and consistent results
- Designed for quick and easy identification
- Ready to use right out of the box; no set-up required
- Ergonomic, rounded shape and no-slip ribbing on grip make lancets easy to handle
- Available in a range of sizes to meet specific blood sample needs
- Not made with natural rubber latex



Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

Never Compromise on CareSM

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.


Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

Haemolance[®] is a registered trademark of HTL-Strefa S.A.


© 2016 McKesson Medical-Surgical Inc.

MCKESSON


PREVENT® HP

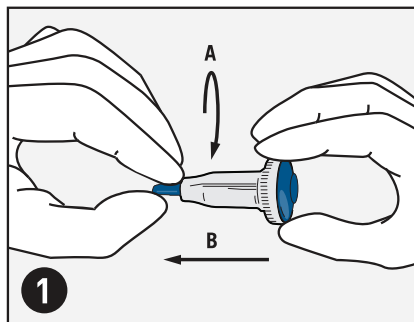

Twist off the protective cap and pull it straight out.

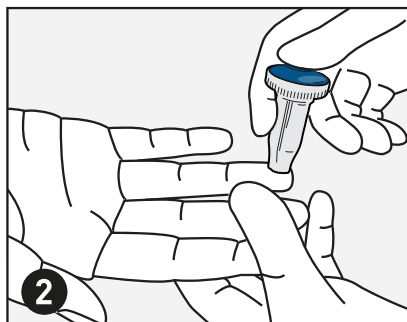

Press the lancet firmly against the puncture site and press the push-button to activate the device. Discard used lancets in accordance with facility guidelines, national and state regulations.

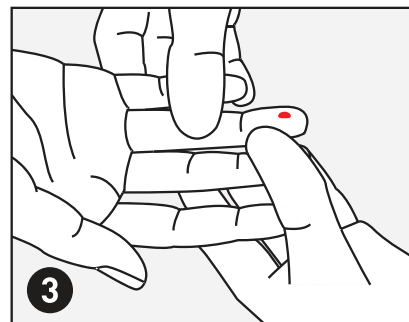






Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

DESCRIPTION

Color

MFR#

QTY

Prevent® HP Push-Button Lancet, 21G x 1.8mm		16-PBHPSL21G1.8	100ea/bx, 20bx/cs
Prevent® HP Push-Button Lancet, 25G x 2.0mm		16-PBHPSL25G2.0	100ea/bx, 20bx/cs
Prevent® HP Push-Button Lancet, 25G x 1.4mm		16-PBHPSL25G1.4	100ea/bx, 20bx/cs
Prevent® HP Push-Button Lancet, 28G x 1.6mm		16-PBHPSL28G1.6	100ea/bx, 20bx/cs
Prevent® HP Push-Button Lancet, 28G x 1.0mm		16-PBHPSL28G1.0	100ea/bx, 20bx/cs