



Premium European

Ankle Foot Orthosis

Push[®] ortho Ankle Foot Orthosis (AFO)

Recreates natural gait

- Offers dynamic dorsiflexion during swing phase and controls plantar flexion through stance phase
- Allows natural forefoot rolling
- Offers medial / lateral support, controlling inversion / eversion
- Secures foot in orthosis, allowing variety of footwear
- Serves as night splint, if desired
- Must be custom fitted by a trained professional

Indications

- Drop foot with intact plantar flexion

Contraindications

- Loss of plantar flexion
- Moderate to severe spasticity
- Extreme foot deformity



Assigned L-code L1930

Sizing: Right or Left Ankle

SIZE	INCHES	CENTIMETERS
1	10 1/2 - 12 1/4	27 - 31
2	12 1/4 - 13 3/8	31 - 34
3	13 3/8 - 15 3/4	34 - 40



“The Push AFO’s unique design enables it to perform several functions. The relatively supple sole leaves dorsal flexion in the ankle joint free and enables the (residual) function of the calf muscles during push-off to contribute to an efficient and natural gait pattern. At heel strike it has a restraining function on the plantar flexion of the foot, and at stance phase it gives bilateral support to the ankle joint.”

— E.P.F. Janssen, MD
Zuyderland Medical Centre, the Netherlands

Safe and Efficient Walking



Foot flat

Controlled plantar flexion through elastic straps



Mid stance

Stable ankle via frame and strap support



Heel off

Forefoot rolling with flexible footplate



Swing phase

Dynamic dorsiflexion through elastic straps



Heel strike

Stable heel strike with a neutral foot position

Premium Comfort & Care

Push braces are designed in consultation with medical experts to be anatomically correct and provide optimal support with minimal impact on daily activities. They are made in Europe with premium quality, machine-washable materials for comfort and durability, and have a 90-day guarantee. Most Push products are made of Sympress™, a soft, moisture-wicking microfiber.

As the exclusive US distributor of Push braces, BraceLab is dedicated to customer satisfaction. We offer exceptional customer service with rapid response and trained support specialists 9am - 5pm EST, Monday - Friday.



Ordering Push® Braces

- Order via website or phone with credit card
- Ask about our volume discounts, or how to establish purchase order/billing account

To help your patients order:

Because the Push ortho Ankle Foot Orthosis requires customization for proper fit, we highly recommend that you order and fit the brace for your patients. If your patients desire to order directly, they must call us at 888-235-8221. We suggest that you:

- Help them determine the correct size
- Let them know they must file their own insurance; BraceLab is unable to assist
- Make them aware that we cannot allow returns or exchanges if the brace has been cut or altered in any way