# Raised Toilet Seat with Armrests

	NEW CODE	Maximum User Weight
Raised Toilet Seat with Armrests	09 155 8873	

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

**Manufactured for  
Performance Health International Ltd.**

Nunn Brook Road, Huthwaite,  
Sutton-in-Ashfield, Nottinghamshire,  
NG17 2HU, UK.

**UK:**

Tel: 03448 730 035  
Fax: 03448 730 100  
www.performancehealth.co.uk

**International:**

Tel: +44 1623 448 706  
Fax: +44 1623 448 784

**Performance Health Supply, Inc.**

28100 Torch Parkway Suite 700,  
Warrenville, IL 60555-3938, USA.  
Tel: 1-800-323-5547  
Telefax: 1-800-547-4333

**Performance Health Australia**

Unit 3, 3 Basalt Road,  
Pemulwuy NSW, Australia 2145.  
Tel: 1300 473 422  
Fax: 1300 766 473

**Performance Health France**

Zone Industrielle de Montjoly,  
11 Avenue de l'Industrie - CS 40712  
08013 Charleville-Mézières Cedex, France.  
Tel: 03 24 52 61 15  
Fax: 03 24 52 90 34

# Raised Toilet Seat with Armrests

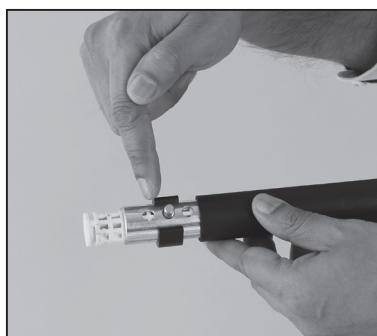
Fig. 1

Fig. 2

Fig. 3

Fig. 4a

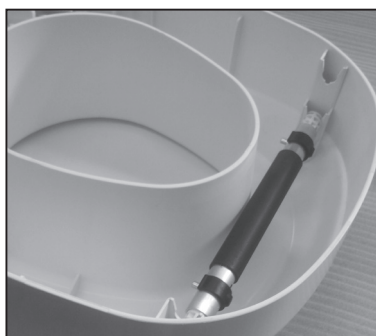
Fig. 4b

Fig. 5a

Fig. 5b

Fig. 6

Fig. 7a

Fig. 7b

Fig. 8

Fig. 9

Dimensions	
Overall Width	500mm (19½")
Overall Depth	475mm (18⅔")
Overall Height	380mm (15")
Seat Width	360mm (14")
Seat Depth	425mm (16⅔")
Aperture Size	210×275mm (8¼×10⅔")
Weight	4.3kg

### Intended uses

- Toilet aids allow for, or increase, independence when using the toilet, particularly by compensating for impaired movement.

### Technical Information

- Maximum user weight – 150kg / 330lb / 23½st
- Seat height – 60mm, 100mm and 150mm

### Instructions for installation

**\*\*Before installing the raised toilet seat, decide on the required position of the seat. The seat has three positions to provide the most appropriate position and comfort for the user. The ideal seat height is when the users feet can completely touch the floor, and the hip height is higher than the knee height i.e. less than 90 degrees on the knee joint or hip joint\*\***

1. Flip up the raised toilet seat for easier access to the adjustment bars.
2. The adjustment cross bar adjusts in width for each seat position. There are three holes at each side of the cross bar to adjust the width (**see fig.1**). For the 60mm height, insert the e-clips into both inner holes of the front bar. For the 100mm height, insert the e-clips into both middle holes of the front bar. For the 150mm height, insert the e-clips into both outer holes of the front bar.
3. Remove the e-clips from the cross bar and confirm the correct width for each seat position. (**see fig.2**)
4. Once confirmed, push the e-clips back into the cross bar. (**see fig.3**)
5. Insert the cross bar under the seat and locate the ends of the cross bar into the locators until they are secure. (**see fig.4a and 4b**)
6. Remove both black knobs at the rear sides of the raised toilet seat.
7. Position the rear bar at the same height of the front bar. For the 60mm height, set both bars at the top locators. For the 100mm height, set both bars to the middle locators. For the 150mm height, set both bars at the bottom locators.
8. Insert both black knobs back through the holes, aligning with the holes of the rear bar and tighten until firm.

### Installing the raised toilet seat

9. Seats are generally attached by a pair of bolts/nuts located under the rear rim of the toilet seat. Unscrew and remove the existing fixings to remove the original toilet seat.
10. Ensure the toilet is clean before installing the raised toilet seat with armrests.
11. Unscrew and remove the long plastic nuts from the rear of the raised toilet seat with armrests.
12. Utilising the holes at the rear of the toilet basin, insert the rear securing bolts. Ensure the seat is centred over the toilet basin.
13. Once the securing bolts are in position (**see fig.5a and 5b**), thread the plastic nut into position with the smooth side facing upwards.
14. Screw the white plastic securing nuts and tighten until secure. (**see fig.6**)

### Installing the armrests into the raised toilet seat

15. With the armrest in the downward position and whilst pushing down on the plastic push pin, align one of the armrests into the hole at the side of the seat and insert the armrest. (**see fig.7a and 7b**)
16. Once the armrest has been inserted fully, rotate the armrest until its normal resting position. (**see fig.8**)
17. The metal stopper on the rest should be resting on the plastic ledge of the toilet seat (**see fig.9**). This provides additional stability and strength. Ensure the armrests are stable and secure before each use.

18. Repeat this process on the other side of the seat to complete the installation.

### Care and cleaning

To clean the raised toilet seat with armrests, wipe with a mild soap and warm water solution. Rinse well and then dry thoroughly with a clean, dry cloth. For heavier soiling, wipe clean with a diluted disinfectant solution and rinse well before drying thoroughly. Do not use water with a temperature in excess of 80°C. Do not use high pressure cleaners, steam cleaners or harsh, abrasive cleaning solutions.

### Cautions

- A full risk assessment should be carried out before use, of the user and the raised toilet seat to ensure both are suitable.
- The toilet seat is not to be used to support more than 150kg / 330lb / 23½st.
- The armrests have not been designed to take the full weight of the user. Before each use, ensure the armrests are stable and secure.
- Do not hold onto the toilet seat as a support when getting on and off the seat.
- Be sure the toilet seat and armrests are securely in place before putting weight on it.
- Regularly check the rear securing bolts are tight and that the raised toilet seat is secure to the toilet basin.
- Periodically, check the raised toilet seat with armrests for any wear and damage.
- Under no circumstances should you stand on the raised toilet seat or armrests.
- This raised toilet seat with armrests is not compatible with all toilets (w/c) on the world market. Ensure that the raised toilet seat is compatible with the toilet before installation and use.

### Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of appropriately.

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