



Walker Trays

Item #A772101—Standard
Item #081554930—Bariatric

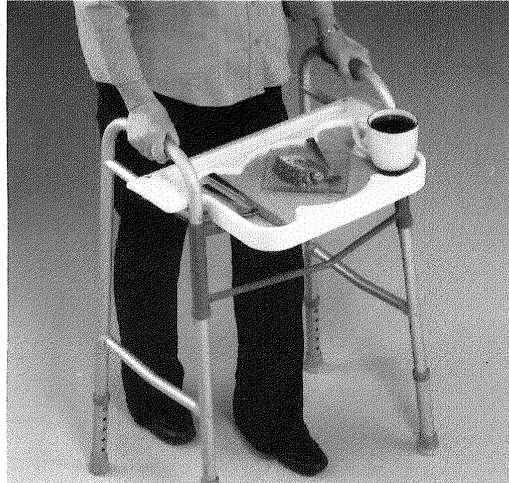

DESCRIPTION:

The Sammons Preston Walker Tray's primary use is for transport of food and/or beverages (hot or cold) from one place to another on your walker. The blue vinyl insert is to be used in the intended area to create traction between the plate and the tray. It is made of durable material and comes fully assembled. It is lightweight for ease of use. The trays can be used with a standard or wheeled walker.

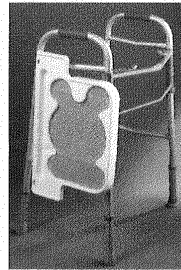

INSTRUCTIONS:

- Lay the tray on top of your walker
- Press down to secure tray to walker
- To adjust position of tray, lean against it to push forward for proper walking position

CLEANING INSTRUCTIONS:

Tray can be washed in dishwasher—top rack only



Sammons Preston
A Patterson Medical Company
1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310



Sammons Preston
A Patterson Medical Company
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012



Authorized European Representative
Homecraft Rolyan
A Patterson Medical Company
Nunn Brook Road, Huthwaite
Sutton-In-Ashfield, Nottinghamshire NG17
2HU
United Kingdom
Phone: 08702 423 305
Fax: 08701 660 211

International
Phone: +44 1623 759 146
Fax: +44 1623 726 886

© November 2010
Patterson Medical Products, Inc.