

## A unique approach to wound management

Triad is a sterile coating that can be applied anywhere, adheres to wet skin, and keeps the wound covered.



Wounds on **irregular surfaces**



**Broken skin** in the presence of incontinence



**Maceration** of peri-wound skin




**Necrotic tissue** such as slough or eschar


### Triad application:



1. Always cleanse the wound before applying Triad.

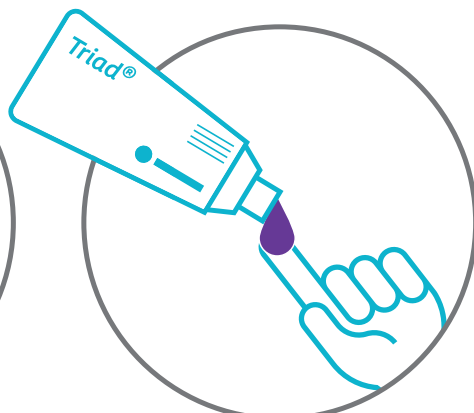

2. Triad can be applied directly from the tube or by using a gloved finger.

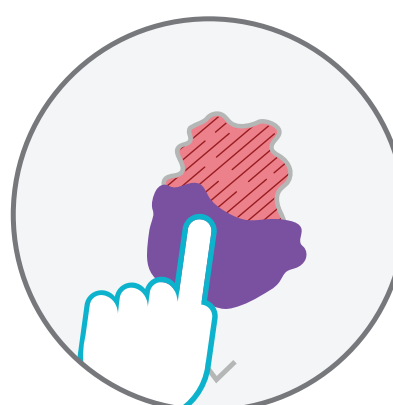


3. Gently spread Triad evenly over the area of application to the thickness of a dime.


Because Triad is designed for use on wet and irregular surfaces it is **typically used on its own without a secondary dressing.**

Triad may not always dry completely and some light staining on bed linens or garments might occur.


### Triad on wounds with depth:

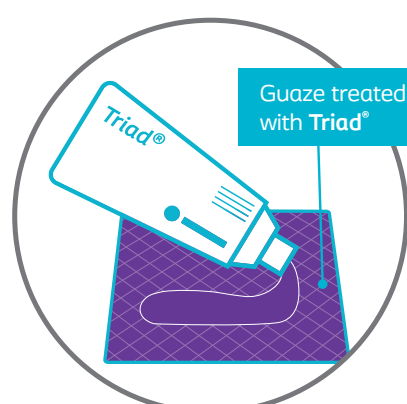

1. For wounds with depth, gauze impregnated with Triad may be used.

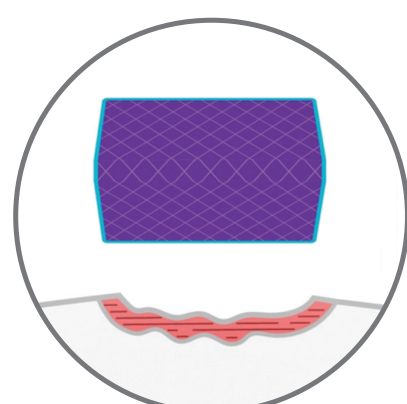

2. Fold or fluff the gauze to fit within the wound bed.

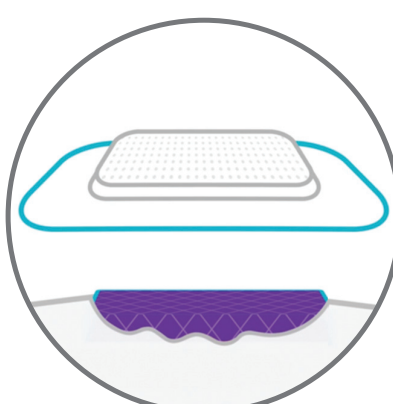

3. Cover the wound with a secondary dressing.

### Triad instructions for use video:


Scan or text below to view the video:

Scan QR code using your smart phone camera. Launch video by tapping the link that appears on your screen (not compatible with all smart phones).



Or, text **94253** and enter keyword **"triad"** to view video.


### Triad removal:

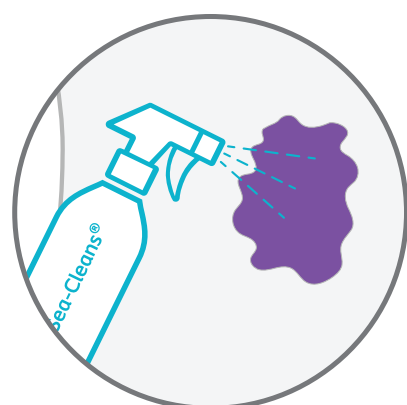

1. Use pH-balanced wound cleanser to soften Triad.



2. Gently wipe to remove without scrubbing.

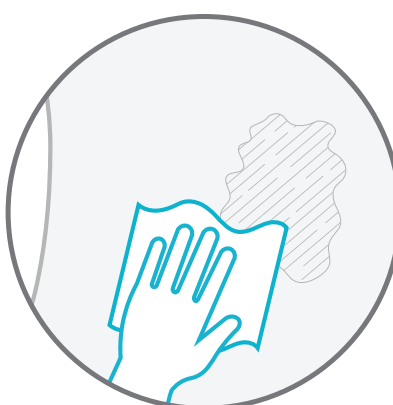

3. For complete removal repeat as needed.

Triad can stay in place for **5-7 days**, with higher exudate levels requiring more frequent re-applications.

In the **perineal area**, reapply Triad after each episode of incontinence.

### Ordering Information

#### Triad<sup>®</sup> Hydrophilic Wound Dressing

Code	Size	Units	HCPS Code
1964	2.5 oz. (71 g)	12	A6240
1967	6 oz. (170 g)	12	A6240



#### Sea-Cleans<sup>®</sup> Wound Cleanser

A pH balanced wound cleanser that can be used to remove Triad without scrubbing.

Code	Size	Units	HCPS Code
1063	6 fl. oz. (178 mL)	12	A6260
1067	12 fl. oz. (355 mL)	12	A6260



Triad is only sterile on first application.

Ostomy Care / Continence Care / Wound & Skin Care / Interventional Urology

Coloplast United States  
Minneapolis, N 55411 / 1-800-533-0464  
[www.coloplast.us](http://www.coloplast.us)

Coloplast Canada  
Oakville, ON L6H 5S9 / 1-877-820-7005  
[www.coloplast.ca](http://www.coloplast.ca)

The Coloplast logo is a registered trademark of Coloplast A/S.  
© 2020 Coloplast Corp. All rights reserved.